Silk Line

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English

Contents

Page	Chapter
27	Information for the user
27	Symbols
28	Identification of warnings
29	Safety precautions
32	Product description
34	Purpose
35	Technical data
37	Transport and storage conditions
37	Use
43	Hygiene and maintenance
46	Disposal
47	Technical Assistance
47	Certification
48	Warranty

Information for the user

Dear Customer,

Thank you for purchasing this product from Ivory. Your new product is designed and made to the highest standards to ensure excellent performance and easy operation and maintenance. Please read the operation manual thoroughly before using the product, and keep the manual where it is readily available to all users of the product.

This document is intended for medical personnel, dentists and dental assistants.

Symbols

	Danger symbol.
	Warning symbol.
	Caution symbol.
	Hint symbol.
	Read usage instructions symbol.
	CE Mark (European Community). Products bearing this mark conform to applicable EU Directives.

	Class II medical device
	Applied Part: Type B
	Manufacturer symbol.
	This product is Autoclavable.
	This product can be washed via Thermo Disinfector.
	Do not dispose of with domestic waste.
	Request for assistance

Identification of warnings

Pay particular attention to the following warnings in order to avoid injury to persons and damage to equipment.

DANGER!

Indicates a risk that can cause death or serious injury if not avoided.

WARNING!

Indicates a risk that can cause serious injury or moderate injury if not avoided.

CAUTION!

Indicates a risk that can cause damage to equipment or moderate or slight injury if not avoided.

HINT!

General explanations, without risk to persons and/or objects.

Safety precautions

WARNING!

In the event of lesions to soft tissue in the oral cavity, or recent tooth extractions, do not perform treatments using instruments powered by compressed air. Infected substances could enter the lesions and cause infection. There is also a risk of air embolism.

CAUTION!

After treatment, remove the bur or stone before placing the handpiece back in its support. Take care not to drop!

WARNING!

Stop using the handpiece immediately if you notice unusual functioning, excessive vibrations, overheating or any other indications of a malfunction in the device.

WARNING!

Do not use this medical device as a light probe. Rotating burs or stones can cause oral lesions.

CAUTION!

Clean, service and dry the instrument as instructed before extended periods of disuse.

CAUTION!

Always perform regular functioning checks and maintenance even after the warranty has expired. The following personnel are authorised to undertake the repair and maintenance of this Ivory product:

- Technicians working in Ivory factories around the world.
- Technicians who have completed Ivory technical training.

Technical assistance must only be performed by qualified Ivory technicians, who can also provide original Ivory spare parts.

 CAUTION!

Do not attempt to disassemble the product or tamper with any mechanism. This medical product must be used in the ways illustrated in this manual. The products and systems specified in the manual for cleaning and maintenance must also be used. Users are responsible for regular functioning checks, maintenance and inspection of the product.

 WARNING!

Sterilise the handpiece and all accessories before use.

 CAUTION!

Dirty or humid cooling air can cause malfunctioning and rapid bearing wear. Cooling air must be clean, dry and uncontaminated, and must conform to ISO 7494-2 standard.

 CAUTION!

Only use clean, undamaged burs to prevent dirt from penetrating into the locking system.

 DANGER!

A bur that is loose, unsuitable or improperly seated can work free of the head or break. Before use always confirm that the bur is correctly seated and locked into place.

 CAUTION!

Insufficient cooling can lead to overheating of the preparation and even damage the tooth. Make sure that the water flow rate is greater than 50NL/min.

 WARNING!

Do not use the handpiece to move the patient's cheek while the device is functioning!

 WARNING!

Do not look at the LED light for a long time when it is illuminated.

 WARNING!

Always check the handpiece for damage before using it.

 CAUTION!

Check the supply pressure before using the handpiece. To do so, fit a pressure gauge between the dental cord and the coupling. Check the following pressures:

- Operating air pressure: 2.8 bar (40 psi)
- Return air: < 0.5 bar (7 psi)
- Water: from 0.8 to 2.5 (12 to 36 psi)
- Spray air: from 1.0 to 4.0 (15 to 58 psi)

Do not exceed the above pressures.

 WARNING!

Check that all the O-rings on the coupling are correctly

seated and undamaged before using the handpiece.

 DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

 CAUTION!

Use of unfiltered, hard water can lead to blockages and premature wear in hoses, fittings and spray heads.

Product description

Silk Line turbine handpiece, push button locking, 3 sprays, optic fibre, Kavo® coupling. **REF. T501LK**

Silk Line turbine handpiece, push button locking, 3 sprays, optic fibre, NSK® coupling. **REF. T501LN**

Silk Line turbine handpiece, push button locking, 3 sprays, optic fibre, Sirona® coupling. **REF. T501LS**

Silk Line turbine handpiece, push button locking, 3 sprays, optic fibre, BienAir® coupling. **REF. T501LB**

Silk Line turbine handpiece, push button locking, 4 sprays, no optic fibre, Borden® 2-way coupling. **REF. S502B**

Silk Line turbine handpiece, push button locking, 4 sprays, no optic fibre, Midwest® 4-way coupling. **REF. S504M**

Purpose

This medical device is designed exclusively for the treatment of dental lesions and pathologies in a dental clinic, and conforms to applicable national legal standards. All other uses and modifications to the product are strictly forbidden, and can lead to hazardous situations.

This medical product is intended for the following applications:

- the removal of dental caries;
- the preparation of cavities and crowns;
- the removal of old fillings;
- the reconstruction and finishing of tooth surfaces.

All uses of this device other than those intended, and the use of accessories and spare parts not authorised by Ivory is strictly forbidden and can even be dangerous.

Failure to conform to the instructions provided herein

invalidates the warranty and exonerates the manufacturer from any other claims. This product must only be used by specialist dental personnel, in a dental clinic or laboratory. In conformity to these requirements, this product must only be used by personnel specialising in its intended uses. The following requirements must be respected:

- provisions governing safety in the workplace;
- provisions governing accident prevention;
- the instructions provided in this manual.

The user must:

- use only perfectly functional equipment;
- use the product only for its intended purpose;
- protect himself, patients and third parties against potential hazards;
- avoiding contamination by the product.

Technical data

Product code	T501LK	T501LN	T501LS	T501LB	S502B	S504M
Type of head	Standard	Standard	Standard	Standard	Standard	Standard
Optic fibre	Yes	Yes	Yes	Yes	No	No
Light intensity up to (lux)	40,000	40,000	40,000	25,000	/	/
Light source / Voltage (vdc)	Glass Rod	Glass Rod	Glass Rod	LED / 3.3-3.8	/	/
Casing material	Titanium steel	Titanium steel	Titanium steel	Titanium steel	Steel	Steel
Locking mechanism	Button	Button	Button	Button	Button	Button
Type of spray	3 nozzles	3 nozzles	3 nozzles	3 nozzles	4 nozzles	4 nozzles
Type of bearings	Ceramic	Ceramic	Ceramic	Ceramic	Ceramic	Ceramic
Type of coupling	Kavo®	NSK®	Sirona®	BienAir®	Borden®2-way	Midwest®4-way
Maximum rotation speed (rpm)	345,000	345,000	345,000	345,000	345,000	345,000
Power (W)	23	23	23	23	23	23

Product code	T501LK	T501LN	T501LS	T501LB	S502B	S504M
Air pressure (bar)	2.8	2.8	2.8	2.8	2.8	2.8
Head diameter (mm)	12.5	12.5	12.5	12.5	12.5	12.5
Head height (mm)	13.1	13.1	13.1	13.1	13.1	13.1
Handpiece weight (g)	44	48	44	56	56	58
Bur diameter (mm)	1.59-1.60	1.59-1.60	1.59-1.60	1.59-1.60	1.59-1.60	1.59-1.60
Bur length (mm)	10-25	10-25	10-25	10-25	10-25	10-25
Type of lubrication	Oil/grease	Oil/grease	Oil/grease	Oil/grease	Oil/grease	Oil/grease
Autoclave resistant	Yes, 135°C	Yes, 135°C	Yes, 135°C	Yes, 135°C	Yes, 135°C	Yes, 135°C
Packing size lxdxh (mm)	160x90x43	160x90x43	160x90x43	160x90x43	160x90x43	160x90x43
Packing weight appr. (g)	250	250	250	250	250	250
Certification	CE 0476	CE 0476	CE 0476	CE 0476	CE 0476	CE 0476

Transport and storage conditions

	Temperature: from -40 °C to +70 °C (from 40 °F to +158 °F).
	Relative humidity: from 10 % to 80 % non-condensing.
	Atmospheric pressure: from 500 hPa to 1060 hPa (from 7 to 15 psi).
	Protect against humidity.
	Fragile.

Use

CAUTION!

Read the safety precautions before starting to use the instrument.

CAUTION!

At the start of every working day and after the treatment of every patient, the water-conducting systems should be rinsed for at least 2 min. without the instrument being attached.

CAUTION!

Do not disconnect the handpiece from the coupling while it is functioning.

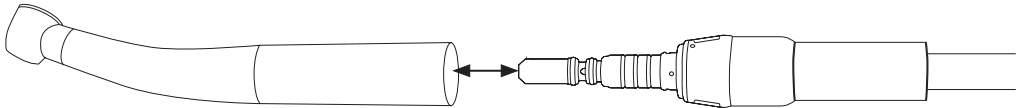
DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.


Fitting and removing the turbine handpiece (Kavo®, Sirona®, Nsk®, BienAir® couplings)

- 1) Push the turbine handpiece into the coupling until it clicks into place.
- 2) Hold the coupling and cord ring nut steady while turning the turbine handpiece gently to remove it.
Never pull on the cord or exert force on it.


- Kavo® coupling


- Sirona® coupling

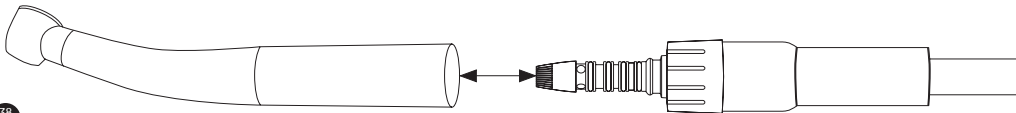- Nsk® coupling

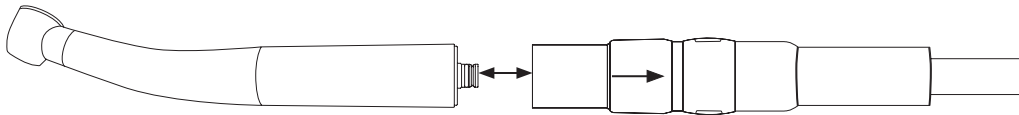
- BienAir® coupling

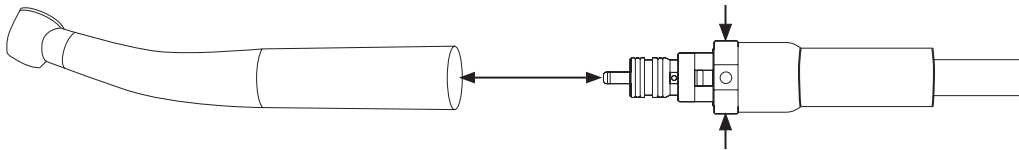
Fitting and removing the turbine handpiece (Borden® and Midwest® couplings)

- 1) Align the turbine handpiece and the Borden®/Midwest® coupling with the holes in the cord. Screw on the cord's ring nut.
- 2) Unscrew the cord's ring nut to remove the turbine handpiece.

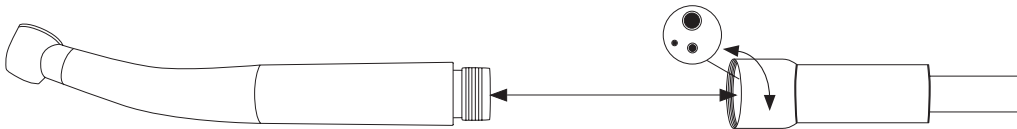- Borden® coupling

- Midwest® coupling

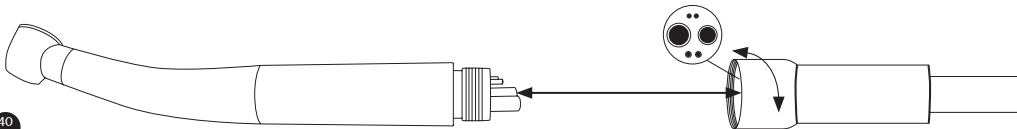Fitting and removing the rotating bur

WARNING!

Never use burs that are bent, deformed, worn, corroded, broken or defective, burs with cracks in the edges or shafts, burs that do not conform to ISO standards, burs that have been tampered with, or burs that do not conform to the specifications given herein. Follow the instructions provided by the manufacturer of the bur you are using. Only use burs that satisfy the following criteria: Bur shaft diameter: 1.59/1.60mm, type 3, conforming to ISO 1797-1, length from 10 to 25 mm.

DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

DANGER!

Only fit the bur in the handpiece when the handpiece is stopped. Never attempt to grip a bur while it is rotating, even when it is nearly stopped.

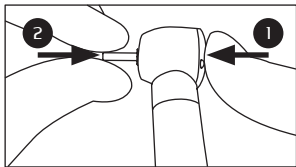
DANGER!

Never press the locking button on the handpiece while the instrument is in use or has not quite stopped. Doing so can cause the bur to become loose or cause the locking system to overheat and break.


DANGER!

AVOID CONTACT between the patient's cheek and the locking button during use. This could lead to the bur being accidentally released, resulting in burns or lesions to the patient.


1) Press the locking button (1) and push the bur (2) in as far as it will go.

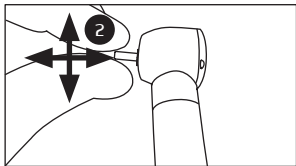2) Check that the bur (2) is held firmly in place.

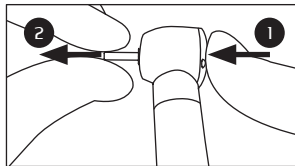3) Press the locking button (1) to release the bur (2).

DANGER!

If you notice any unusual noise, vibration, overheating, interruption in the flow of cooling liquid, leakage or changes in the colour of the LED, stop using

Hygiene and maintenance

WARNING!

Respect all directives, standards and national legislation governing cleaning, disinfection and sterilisation.

DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

CAUTION!

Never immerse the handpiece in disinfectant solutions or ultrasonic baths! Only use chlorine-free disinfectant products that have been certified by officially recognised institutes.

-External disinfection

Clean and disinfect the handpiece after every treatment to eliminate residues of bodily fluids (e.g. blood, saliva etc.) and

prevent them entering the internal parts of the handpiece.

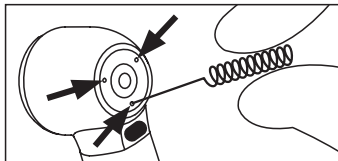
1) Clean off all dirt and impurities from the handpiece. Do not use metal brushes.

2) Clean the handpiece with disinfectant wipes.

3) Blow the handpiece dry with a jet of compressed air.

- Cleaning the spray nozzles

Use the cleaning wire provided to gently remove dirt and deposits from the spray nozzles. Clean and disinfect the cleaning wire.


! CAUTION!

Do not force the brush into the conduit. Damage to the conduit can change the direction of spray with respect to the bur and reduce the effectiveness of cooling.

- Cleaning the optic fibre glass (if fitted)

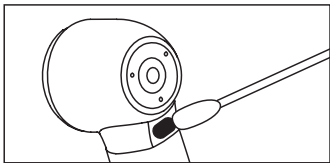Clean the glass of the optic fibre with a pad damped in alcohol. Remove all residual impurities.

- Lubrication

! CAUTION!

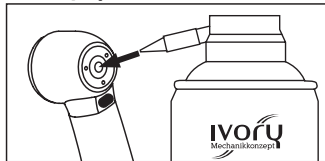
Lubrication cycles:

- After every 15 minutes of rotation.
- Before every sterilisation.

! CAUTION!

Lubricate only with Ivory oil OL101S.

1) Fit the universal Ivory turbine handpiece adapter to the oil spray can.

2) Insert the other end of the universal Ivory adapter in the hole in the back of the handpiece spindle. Hold the handpiece in place. 3) Spray for about 5 seconds.

CAUTION!

After lubricating, hold the handpiece with the chuck facing down. Operate the handpiece for 30 seconds to expel any excess oil. If you notice any dirt coming out, repeat the entire cleaning and lubrication process.

-Sterilisation

Sterilisation in an autoclave conforming to: EN 13060, EN ISO 17665-1

CAUTION!

- Follow the instructions of the autoclave manufacturer.
- Clean, disinfect and lubricate the handpiece before sterilisation. The presence of blood residues on the external or internal surfaces can lead to the formation of corrosion.
- Make sure that the handpiece is properly dried after sterilisation.

Authorised sterilisation procedures

- 1) Seal the handpiece and its accessories in sterilisation bags conforming to EN 868-5.
- 2) Sterilise the bagged handpiece in an autoclave according to **EN ISO 17665-1** standard and respect the following conditions:
 - Class B steam sterilisation in autoclaves conforming to **EN 13060** standard. Sterilisation time at least 3.5 minutes at 135 °C.
- 3) Remove the bag from the autoclave.

-Storage

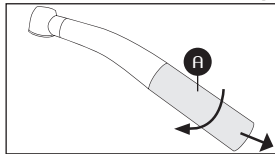
- Keep the handpiece in the sterilisation bag until it is needed.
- Once sterilised, store the product in a dry and dust-free place, if possible with a low microbial charge.

CAUTION!


Respect the validity date for the sterilisation!

-Replacing the LED (Model: REF. T501LB)

1) Unscrew the back (A) of handpiece sleeve and remove it.

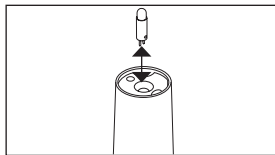2) Remove the old LED and fit the new one.

3) Screw the handpiece sleeve back on.

HINT!

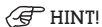
If the new LED fails to light, remove it and rotate it through 180° to invert polarity.

Disposal

Sterilise the product and its accessories before disposing of them. Refer the disposal of the product  to authorised companies qualified to dispose of special industrial waste.

Technical assistance – Ivory direct line

HINT!

To request technical assistance, compile the on-line form available at: www.ivoiry-dent.com

Declaration of conformity

This medical device conforms to the requirements of European Directives 93/42 CEE (Class IIa) and 2007/47/CE.
This conformity is expressed by the mark:

Additional documentation for controller n°0476 can be requested from: ivoiry@ivoiry-dent.com

Warranty

Ivory guarantees this product against defects of material and manufacture for a period of 24 months from the date of purchase. Ivory cannot be held responsible for defects and consequences caused by or attributable to normal wear, improper use, cleaning or maintenance, failure to follow operating and coupling instructions, lime scale or corrosion, contamination of the air or water supplies, chemical or electrical effects of an exceptional nature or not permitted by the instructions provided by Ivory or other manufacturers. Ivory expressly declines all responsibility for defects or consequences attributable to tampering with the product or modifications made by the customer or unauthorised third parties, or attributable to the use of non-original spare parts. In the event of justified claims, Ivory shall either repair the product or supply spare parts free of charge under the warranty. The warranty excludes all other claims, with particular reference to claims for compensation. In the case of delays, serious or wilful negligence, this exclusion is only valid provided no specific laws are contravened.

The warranty does not apply to lamps, glass and optic fibre conductors, glass or rubber parts, or the colour fastness of plastic materials or all parts subject to normal wear. Claims made under the warranty must be presented by compiling the on-line request "QRRMA" form and by accompanying the product with a copy of the purchase invoice.

The purchase invoice must clearly identify the dealer, date of purchase, code or model of the product and the production or serial number. Please refer also to the articles 17,18 and 19 in the "Terms and payment" section on www.ivory-dent.com

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Turbine Silk Line, bouton-poussoir, 3 sprays, fibre optique, raccord Kavo®. **REF. T501LK**

Turbine Silk Line, bouton-poussoir, 3 sprays, fibre optique, raccord NSK®. **REF. T501LN**

Turbine Silk Line, bouton-poussoir, 3 sprays, fibre optique, raccord Sirona®. **REF. T501LS**

Turbine Silk Line, bouton-poussoir, 3 sprays, fibre optique, raccord BienAir®. **REF. T501LB**

Turbine Silk Line, bouton-poussoir, 4 sprays, sans fibre optique, raccord 2 voies Borden®. **REF. S502B**

Turbine Silk Line, bouton-poussoir, 4 sprays, sans fibre optique, raccord 4 voies Midwest®. **REF. S504M**

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KaVo® is a registered trademark of Kaltenbach & Voigt GmbH & Co., Germany. Sirona® is a registered trademark of Sirona Dental Systems GmbH, Germany. Bien-Air® is a registered trademark of Bien-Air Dental S.A., Switzerland. NSK® is a registered trademark of NSK NAKANISHI INC., Japan. W&H® and Roto Quick® are registered trademarks of W&H Dentalwerk Bürmoos GmbH, Austria.