Silk Line

Luft-mikromotoren / Air micromotors / Micromotori pneumatici
Micromoteurs pneumatiques / Micromotores neumáticos

M714C / M714L

Bedienungsanleitung - Operation manual - Istruzioni per l'uso
Instructions pour l'utilisation - Instrucciones de uso

Deutsch: seite 1
English: page 21
Italiano: pagina 41
Français: page 61
Español: página 81

Deutsch

Inhalt

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English

Contents

Page	Chapter
23	Information for the user
23	Symbols
24	Identification of warnings
25	Safety precautions
28	Product description
29	Purpose
30	Technical data
32	Transport and storage conditions
32	Use
35	Hygiene and maintenance
38	Disposal
39	Technical Assistance
39	Certification
40	Warranty

Information for the user

Dear Customer,

Thank you for purchasing this product from Ivory. Your new product is designed and made to the highest standards to ensure excellent performance and easy operation and maintenance. Please read the operation manual thoroughly before using the product, and keep the manual where it is readily available to all users of the product.

This document is intended for medical personnel, dentists and dental assistants.

Symbols

	Danger symbol.
	Warning symbol.
	Caution symbol.
	Hint symbol.
	Read usage instructions symbol.
	CE Mark (European Community). Products bearing this mark conform to applicable EU Directives.

	Class II medical device
	Applied Part: Type B
	Manufacturer symbol.
	This product is Autoclavable.
	This product can be washed via Thermo Disinfector.
	Do not dispose of with domestic waste.
	Request for assistance

Identification of warnings

Pay particular attention to the following warnings in order to avoid injury to persons and damage to equipment.

DANGER!

Indicates a risk that can cause death or serious injury if not avoided.

WARNING!

Indicates a risk that can cause serious injury or moderate injury if not avoided.

CAUTION!

Indicates a risk that can cause damage to equipment or moderate or slight injury if not avoided.

HINT!

General explanations, without risk to persons and/or objects.

Safety precautions

According to IEC 60601-1 / UL 60601-1, micromotors are not suitable for use in environments at risk of explosion or in environments liable to contain explosive anaesthetic mixes containing oxygen or nitrous oxide.

Use of micromotors with LED illumination is only permitted in conjunction with power supplies conforming to IEC 60601 1 (EN 60601-1) and IEC 60601-1-2 (EN 60601-1-2) provided the following conditions are satisfied and guaranteed by the system installer:

- Double insulation must be provided between the primary and secondary electrical circuits, rated for the maximum foreseen supply voltage.
- Double insulation must be provided between the

secondary circuit and earth (PE), rated for the maximum foreseen secondary voltage.

- Secondary voltage must be limited to a maximum of 3.8 vdc.
- Applied apparatus must satisfy maximum leakage current specifications.
- The secondary electrical circuit must be protected against short circuit and overload.

After treatment, remove the bur or stone before placing the handpiece back in its support. Take care not to drop!

Stop using the handpiece immediately if you notice

unusual functioning, excessive vibrations, overheating or any other indications of a malfunction in the device.

 CAUTION!

Always perform regular functioning checks and maintenance even after the warranty has expired. The following personnel are authorised to undertake the repair and maintenance of this Ivory product:

- Technicians working in Ivory factories around the world.
- Technicians who have completed Ivory technical training.

Technical assistance must only be performed by qualified Ivory technicians, who can also provide original Ivory spare parts.

 CAUTION!

Do not attempt to disassemble the product or tamper with any mechanism. This medical product must be used in the ways illustrated in this manual. The products and systems specified in the manual for cleaning and maintenance must

also be used. Users are responsible for regular functioning checks, maintenance and inspection of the product.

 WARNING!

Sterilise the handpiece and all accessories before use.

 CAUTION!

Dirty or humid cooling air can cause malfunctioning and rapid bearing wear. Cooling air must be clean, dry and uncontaminated, and must conform to ISO 7494-2 standard.

 WARNING!

Do not look at the LED light for a long time when it is illuminated.

 WARNING!

Always check the handpiece for damage before using it.

 CAUTION!

Check the supply pressure before using the handpiece.
To do so, fit a pressure gauge between the dental cord
and the micromotor. Check the following pressures:

- Operating air pressure: 2.8 bar (40 psi)
- Return air: < 0.5 bar (7 psi)
- Water: from 0.8 to 2.5 (12 to 36 psi)
- Spray air: from 1.0 to 4.0 (15 to 58 psi)

Do not exceed the above pressures.

 DANGER!

Always wear gloves to avoid the risk of lesions and
infections when checking, cleaning, repairing or
extracting teeth.

 CAUTION!

Use of unfiltered, hard water can lead to blockages and
premature wear in hoses, fittings and spray heads.

Product description

Air micromotor, internal spray, 4-way coupling.
MOD. M714C

Air micromotor, internal spray, 6-way coupling, with light.
MOD. M714L

Purpose

This medical device is designed exclusively for the treatment of dental lesions and pathologies in a dental clinic, and conforms to applicable national legal standards. All other uses and modifications to the product are strictly forbidden, and can lead to hazardous situations.

This medical product is intended for the following applications:

- the drive of dental handpieces for restorative and prophylactic treatments.

All uses of this device other than those intended, and the use of accessories and spare parts not authorised by Ivory is strictly forbidden and can even be dangerous. Failure to conform to the instructions provided herein invalidates the warranty and exonerates the manufacturer

from any other claims. This product must only be used by specialist dental personnel, in a dental clinic or laboratory. In conformity to these requirements, this product must only be used by personnel specialising in its intended uses.

The following requirements must be respected:

- provisions governing safety in the workplace;
- provisions governing accident prevention;
- the instructions provided in this manual.

The user must:

- use only perfectly functional equipment;
- use the product only for its intended purpose;
- protect himself, patients and third parties against potential hazards;
- avoiding contamination by the product.

Technical data

Product code	M714C	M714L
Version	Air driven	Air driven
Compatibility	4 way	6 way
Light source / Voltage (vdc)	No	Led / 3.3-3.8
Type of spray	Internal	Internal
Maximum rotation speed (rpm)	20,000	20,000
Power	Air	Air
Speed adjustment	Yes	Yes
Reverse rotation	Yes	Yes
Handpiece rotation	Yes	No
Air consumption (NL/min at 0.28MPa)	50	50

Product code	M714C	M714L
Handpiece weight (g)	110	130
Handpiece diameter (mm)	20	20
Type of lubrication	Oil	Oil
Autoclave resistant handpiece	Yes-135°C	Yes-135°C
Autoclave resistant removable shell	No	No
Packing size lxdxh (mm)	160 x 90 x 43	160 x 90 x 43
Packing weight appr. (g)	320	320
Certification	CE 0476	CE 0476

Transport and storage conditions

	Temperature: from -40 °C to +70 °C (from 40 °F to +158 °F).
	Relative humidity: from 10 % to 80 % non-condensing.
	Atmospheric pressure: from 500 hPa to 1060 hPa (from 7 to 15 psi).
	Protect against humidity.
	Fragile.

Use

CAUTION!

Read the safety precautions before starting to use the instrument.

CAUTION!

At the start of every working day and after the treatment of every patient, the water-conducting systems should be rinsed for at least 2 min. without the instrument being attached.

CAUTION!

Do not disconnect the handpiece from the coupling while it is functioning.


DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

Fitting and removing the micromotor

- Align the micromotor with the holes in the cord.
- Screw on the cord ring nut.
- Unscrew the cord's ring nut to remove the micromotor.

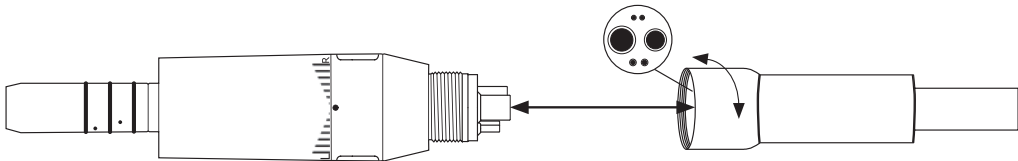Test cycle prior to treatment

DANGER!


If you notice any unusual noise, vibration, overheating, interruption in the flow of cooling liquid, leakage or changes in the colour of the LED, stop using the handpiece and contact an authorised Ivory dealer.

DANGER!

Never attempt to fit an instrument to a rotating micromotor. Always check that the cooling system is unobstructed before starting work.

- 1) Forward rotation: turn the direction ring (1) to "R".
- 2) Reverse rotation: turn the direction ring (1) to "L".
- 3) With the direction ring (1) positioned centrally between "R" and "L", the drive air supply is shut off and the micromotor will not turn.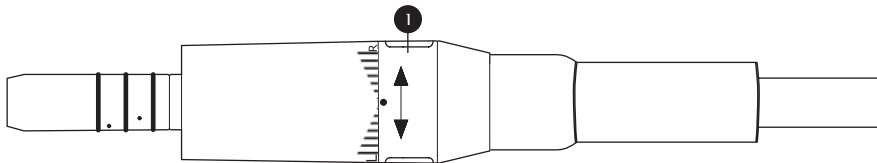

Hygiene and maintenance

WARNING!

Respect all directives, standards and national legislation governing cleaning, disinfection and sterilisation.

DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

CAUTION!

Never immerse the micromotor in disinfectant solutions or ultrasonic baths! Only use chlorine-free disinfectant products that have been certified by officially recognised institutes.

-External disinfection

Clean and disinfect the handpiece after every treatment to eliminate residues of bodily fluids (e.g. blood, saliva etc.) and

prevent them entering the internal parts of the handpiece.


1) Clean off all dirt and impurities from the handpiece. Do not use metal brushes.

2) Clean the handpiece with disinfectant wipes.

3) Blow the handpiece dry with a jet of compressed air.

- Cleaning the spray nozzles

Use the cleaning wire provided to gently remove dirt and deposits from the spray nozzles. Clean and disinfect the cleaning wire.

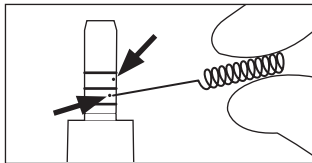CAUTION!

Do not force the brush into the conduit.

- Lubrication

CAUTION!

Lubrication cycles:

- Once a day.
- Before every sterilisation.


CAUTION!


Lubricate only with Ivory oil OL101S.

Legend for holes:

- 1- Air exhaust.
- 2- Drive air inlet.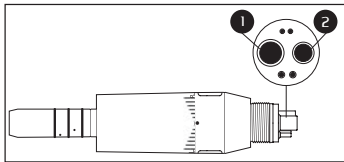

1) Fit the universal Ivory adapter to the oil spray can.

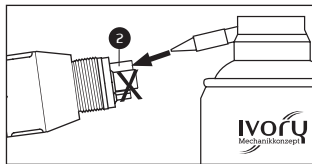2) Insert the other end of the universal Ivory adapter in the drive air inlet (2). Hold the micromotor in place.

3) Spray for about 5 seconds until clean oil flows out from the air exhaust hole (1).

CAUTION!

The ring must be positioned to "R" or "L".

-Sterilisation

Sterilisation in an autoclave conforming to: EN 13060, EN ISO 17665-1

CAUTION!

- Follow the instructions of the autoclave manufacturer.
- Clean, disinfect and lubricate the handpiece before sterilisation. The presence of blood residues on the external or internal surfaces can lead to the formation of corrosion.
- Make sure that the handpiece is properly dried after sterilisation.

Authorised sterilisation procedures

- 1) Seal the handpiece and its accessories in sterilisation bags conforming to EN 868-5.
- 2) Sterilise the bagged handpiece in an autoclave according to **EN ISO 17665-1** standard and respect the following conditions:
 - Class B steam sterilisation in autoclaves conforming to **EN 13060** standard. Sterilisation time at least 3.5 minutes at 135 °C.
- 3) Remove the bag from the autoclave.

-Storage

- Keep the handpiece in the sterilisation bag until it is needed.
- Once sterilised, store the product in a dry and dust-free place, if possible with a low microbial charge.


CAUTION!

Respect the validity date for the sterilisation!

-Replacing the O-rings

Replace the O-rings if you notice any water leaks between the micromotor and the handpiece.


- 1) Squeeze the old O-ring between your fingers to remove it.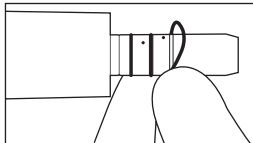


- 2) Fit the new O-ring in the correct seat.

-Replacing the LED (Model: REF. M714L)

1) Unscrew the three screws and gently pull out the top of motor (A).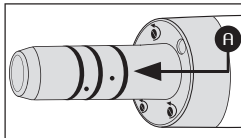

2) Remove the old LED and fit the new one.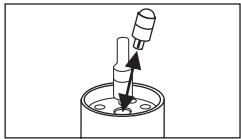

3) Reassemble the motor.


HINT!

If the new LED fails to light, remove it and rotate it through 180° to invert polarity.

Disposal

 Sterilise the product and its accessories before disposing of them. Refer the disposal of the product  to authorised companies qualified to dispose of special industrial waste.

Technical assistance – Ivory direct line

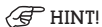HINT!

To request technical assistance, compile the on-line form available at: www.ivory-dent.com

Declaration of conformity

This medical device conforms to the requirements of European Directives 93/42 CEE (Class IIa) and 2007/47/CE.
This conformity is expressed by the mark:

0476

Additional documentation for controller n°0476 can be requested from: ivory@ivory-dent.com

Warranty

Ivory guarantees this product against defects of material and manufacture for a period of 24 months from the date of purchase. Ivory cannot be held responsible for defects and consequences caused by or attributable to normal wear, improper use, cleaning or maintenance, failure to follow operating and coupling instructions, lime scale or corrosion, contamination of the air or water supplies, chemical or electrical effects of an exceptional nature or not permitted by the instructions provided by Ivory or other manufacturers. Ivory expressly declines all responsibility for defects or consequences attributable to tampering with the product or modifications made by the customer or unauthorised third parties, or attributable to the use of non-original spare parts. In the event of justified claims, Ivory shall either repair the product or supply spare parts free of charge under the warranty. The warranty excludes all other claims, with particular reference to claims for compensation. In the case of delays, serious or wilful negligence, this exclusion is only valid provided no specific laws are contravened.

The warranty does not apply to lamps, glass and optic fibre conductors, glass or rubber parts, or the colour fastness of plastic materials or all parts subject to normal wear. Claims made under the warranty must be presented by compiling the on-line request "QRRMA" form and by accompanying the product with a copy of the purchase invoice.

The purchase invoice must clearly identify the dealer, date of purchase, code or model of the product and the production or serial number. Please refer also to the articles 17,18 and 19 in the "Terms and payment" section on www.ivory-dent.com

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KaVo® is a registered trademark of Kaltenbach & Voigt GmbH & Co., Germany. Sirona® is a registered trademark of Sirona Dental Systems GmbH, Germany. Bien-Air® is a registered trademark of Bien-Air Dental S.A., Switzerland. NSK® is a registered trademark of NSK NAKANISHI INC., Japan. W&H® and Roto Quick® are registered trademarks of W&H Dentalwerk Bürmoos GmbH, Austria.