IVORY
Mechanikkonzept

Kupplungen/Couplings/Attacchi
Raccords/Acoplamientos

Bedienungsanleitung - Operation manual
Istruzioni per l'uso - Instructions pour
l'utilisation - Instrucciones de uso

C31K/C30LK/C31LK
C30LW/C30LN/C30LS/C30LB

Deutsch: seite 1
English: page 30
Italiano: pagina 56
Français: page 84
Español: página 112

Deutsch

Inhalt

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English

Contents

Page	Chapter
32	Information for the user
33	Symbols
35	Identification of warnings
36	Safety precautions
38	Product description
40	Purpose
42	Technical data
43	Transport and storage conditions
44	Use
49	Hygiene and maintenance
54	Disposal
54	Technical Assistance
55	Certification
56	Warranty

Information for the user

Dear Customer,

Thank you for purchasing this product from Ivory. Your new product is designed and made to the highest standards to ensure excellent performance and easy operation and maintenance. Please read the operation manual thoroughly before using the product, and keep the manual where it is readily available to all users of the product.

This document is intended for medical personnel, dentists and dental assistants.

Symbols

	Danger symbol.
	Warning symbol.
	Caution symbol.
	Hint symbol.
	Read usage instructions symbol.
	CE Mark (European Community). Products bearing this mark conform to applicable EU Directives.

	<p>Manufacturer symbol.</p>
	<p>This product is not autoclave compatible.</p>
	<p>This product cannot be thermally disinfected.</p>
	<p>Do not dispose of with domestic waste.</p>
	<p>Request for assistance</p>

Identification of warnings

Pay particular attention to the following warnings in order to avoid injury to persons and damage to equipment.

Indicates a risk that can cause death or serious injury if not avoided.

Indicates a risk that can cause serious injury or moderate injury if not avoided.

Indicates a risk that can cause damage to equipment or moderate or slight injury if not avoided.

General explanations, without risk to persons and/or objects.

Safety precautions

WARNING!

Clean the product and store it in a dry place before first putting it into function, after every use and before every extended period of disuse.

CAUTION!

Always perform regular functioning checks and maintenance even after the warranty has expired. The following personnel are authorised to undertake the repair and maintenance of this Ivory product:

- Technicians working in Ivory factories around the world.
- Technicians who have completed Ivory technical training.

Technical assistance must only be performed by qualified Ivory technicians, who can also provide original Ivory spare parts.

CAUTION!

Do not attempt to disassemble the product or tamper with any

mechanism. This medical product must be used in the ways illustrated in this manual. The products and systems specified in the manual for cleaning and maintenance must also be used. Users are responsible for regular functioning checks, maintenance and inspection of the product.

 **WARNING!**

Stop using the handpiece immediately if you notice unusual functioning, excessive vibrations, overheating or any other indications of a malfunction in the device.

 **CAUTION!**

Insufficient cooling can lead to overheating of the preparation and even damage the tooth. Make sure that the water flow rate is greater than 50ml/min.

 **WARNING!**

Do not look at the LED light for a long time when it is illuminated.

 **WARNING!**

Always check the coupling for damage before using it.

Product description

Kavo® Multiflex® compatible coupling, blind, with spray adjustment.

MOD. C31K

Kavo® Multiflex® compatible coupling, LED. **MOD. C30LK**

Kavo® Multiflex® compatible coupling, LED, with spray adjustment.

MOD. C31LK

W&H® Roto Quick® RQ 24
compatible coupling, electrified.
MOD. C30LW

NSK® PTL-CL-LED compatible
coupling, LED. **MOD. C30LN**

Sirona® Click&Go compatible
coupling, LED, with spray
adjustment. **MOD. C30LS**

Bien Air® Unifix compatible coupling,
electrified, with spray adjustment.
MOD. C30LB

Purpose

This medical device is designed exclusively for the treatment of dental lesions and pathologies in a dental clinic, and conforms to applicable national legal standards. All other uses and modifications to the product are strictly forbidden, and can lead to hazardous situations.

This medical product is intended to couple the dental cords to compatible turbine handpieces.

DANGER!

All uses of this device other than those intended, and the use of accessories and spare parts not authorised by Ivory is strictly forbidden and can even be dangerous.

Failure to conform to the instructions provided herein invalidates the warranty and exonerates the manufacturer from any other claims. This product must only be used by specialist dental personnel, in a dental

clinic or laboratory. In conformity to these requirements, this product must only be used by personnel specialising in its intended uses. The following requirements must be respected:

- provisions governing safety in the workplace;
- provisions governing accident prevention;
- the instructions provided in this manual.


The user must:

- use only perfectly functional equipment;
- use the product only for its intended purpose;
- protect himself, patients and third parties against potential hazards;
- avoiding contamination by the product.

Technical data

Product code	C31K	C30LK	C31LK	C30LW	C30LN	C30LS	C30LB
Compatibility	Kavo® Multiflex®	Kavo® Multiflex®	Kavo® Multiflex®	W&H®	NSK®	Sirona®	Bien Air®
Instrument connection							
Light source	No	Led	Led	Yes	Led	Led	Yes
Light intensity (lux)	/	25.000	25.000	/	25.000	25.000	/
360° instrument rotation	Yes	Yes	Yes	Yes	Yes	Yes	Yes
Spray adjustment	Yes	No	Yes	No	No	Yes	Yes
Autoclave resistant	No	No	No	No	No	No	No
Certification	CE	CE	CE	CE	CE	CE	CE

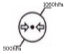Transport and storage conditions

	Temperature: from -40 °C to +70 °C (from 40 °F to +158 °F).
	Relative humidity: from 10 % to 80 % non-condensing.
	Atmospheric pressure: from 500 hPa to 1060 hPa (from 7 to 15 psi).
	Protect against humidity.
	Fragile.

Use

 **CAUTION!**

Read the safety precautions before starting to use the instrument.

 **CAUTION!**

At the start of every working day and after the treatment of every patient, the water-conducting systems should be rinsed for at least 2 min. without the instrument being attached.

 **CAUTION!**

Do not disconnect the handpiece from the micromotor while it is functioning.

 **DANGER!**

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

Fitting and removing the coupling

! CAUTION!

Do not connect a LED coupling to a blind cord.

1) Align the coupling with the holes in the cord.

2) Screw on the cord ring nut.


3) Use the wrench provided  to hold the coupling while tightening

the cord ring nut (only for models C31K / C30LK / C31LK).


4) Unscrew the cord's ring nut to remove the coupling.

Never pull on the cord or exert force on it.


- REF. C31K / C30LK / C31LK / C30LS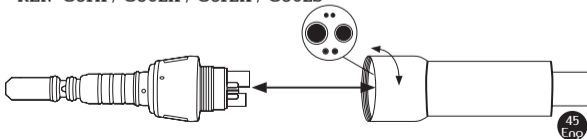


- REF. C30LW

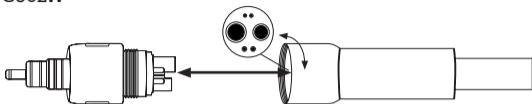- REF. C30LN

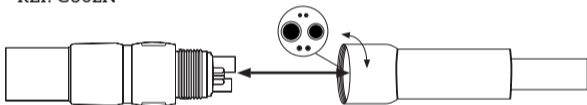- REF. C30LB

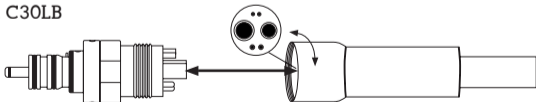 **DANGER!**

If you notice any unusual noise, vibration, overheating, interruption in the flow of cooling liquid, leakage or changes in the colour of the LED, stop using the handpiece and contact an authorised Ivory dealer.


 **DANGER!**

Check that all the O-rings are correctly seated and undamaged before using the coupling. Do not use the coupling if any of the O-rings are missing or damaged.


Fitting and removing the turbine handpiece

- 1) Push the turbine handpiece into the coupling until it clicks into place.
- 2) Hold the coupling and cord ring nut steady while turning the turbine handpiece gently to remove it. Never pull on the cord or exert force on it.


- REF. C31K / C30LK / C31LK / C30LS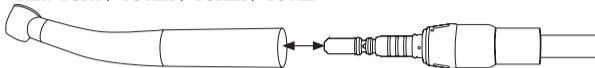


- REF. C30LW

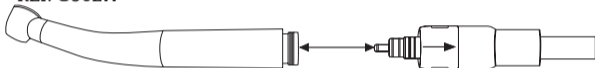- REF. C30LN

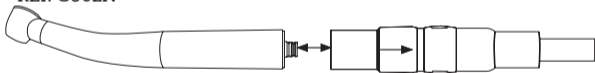
- REF. C30LB

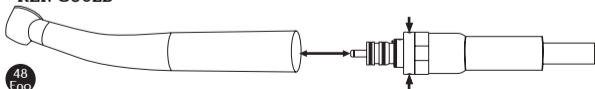Spray adjustment (only on models C31K / C31LK / C30LS/ C30LB)

- 1) Turn the adjuster ring (1) clockwise to reduce the water flow.
- 2) Turn the adjuster ring (1) anti-clockwise to increase the water flow.

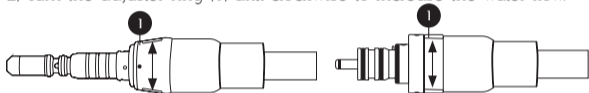Hygiene and maintenance

WARNING!

Respect all directives, standards and national legislation governing cleaning, disinfection and sterilisation.

 **DANGER!**

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

 **CAUTION!**

Never immerse the handpiece in disinfectant solutions or ultrasonic baths! Only use chlorine-free disinfectant products that have been certified by officially recognised institutes.

-External disinfection

Clean and disinfect the coupling after every treatment to eliminate residues of bodily fluids (e.g. blood, saliva etc.) and prevent them entering the internal parts of the coupling.

- 1) Clean off all dirt and impurities from the coupling. Do not clean under running water. Do not use metal brushes.
- 2) Clean the coupling with disinfectant wipes.
- 3) Blow the coupling dry with a jet of compressed air.

- Lubrication

Only lubricate the coupling's O-rings with a pad damped in Ivory oil OL101S.

-Sterilisation

THIS PRODUCT IS NOT AUTOCLAVE COMPATIBLE.

THIS PRODUCT CANNOT BE THERMALLY DISINFECTED.

-Storage


Once sterilised, store the product in a dry and dust-free place, if possible with a low microbial charge.

-Replacing the O-rings

(only on models C31K / C30LK / C31LK / C30LW / C30LS / C30LB)

 **HINT!** Coupling C30LN does not have any O-rings. The O-rings are located in the handpiece fitting.

Replace the O-rings if you notice any water leaks between the coupling and the handpiece.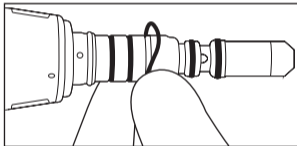

- 1) Squeeze the old O-ring between your fingers to remove it.
- 2) Fit the new O-ring in the correct seat.

-Replacing the LED

- Models: MOD. C30LK / C30LK / C31LK / C30LS

- 1) Unscrew the cap anti-clockwise to remove it.

2) Remove the old LED from the coupling and fit the new one.

 **HINT!** If the new LED fails to light, remove it and rotate it through 180° to invert polarity.


- **Model: MOD. C30LN**

1) Unscrew the ring nut clockwise to remove it.

2) Remove the old LED from the coupling and fit the new one.

 **HINT!** If the new LED fails to light, remove it and rotate it through 180° to invert polarity.

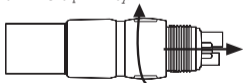Disposal

Sterilise the product and its accessories before disposing of them.
Refer the disposal of the product to authorised companies
qualified to dispose of special industrial waste.

Technical assistance – Ivory direct line

HINT! To request technical assistance, compile the on-line form
available at: www.ivory-dent.com

Declaration of conformity

This medical device conforms to the requirements of European Directives 93/42 CEE (Class I) and 2007/47/CE.

This conformity is expressed by the mark:

Additional documentation can be requested from: *ivory@ivory-dent.com*

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KaVo® is a registered trademark of Kaltenbach & Voigt GmbH & Co., Germany. Sirona® is a registered trademark of Sirona Dental Systems GmbH, Germany. Bien-Air® is a registered trademark of Bien-Air Dental S.A., Switzerland. NSK® is a registered trademark of NSK NAKANISHI INC., Japan. W&H® and Roto Quick® are registered trademarks of W&H Dentalwerk Bürmoos GmbH, Austria.