# Silk Line

Winkelstücke/Contra-angles/Contraangoli  
Contre-angles/Contraángulos

SL411C-SL411L-SL421C-SL421L-SL415C-SL415L-SL4201C

Bedienungsanleitung - Operation manual - Istruzioni per l'uso  
Instructions pour l'utilisation-Instrucciones de uso

Deutsch: seite 1  
English: page 29  
Italiano: pagina 57  
Français: page 85  
Español: página 113

# Deutsch

## Inhalt

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# English

## Contents

Page	Chapter
31	Information for the user
31	Symbols
32	Identification of warnings
33	Safety precautions
36	Product description
40	Purpose
41	Technical data
43	Transport and storage conditions
43	Use
50	Hygiene and maintenance
54	Disposal
55	Technical Assistance
55	Certification
56	Warranty

## Information for the user

Dear Customer,

Thank you for purchasing this product from Ivory. Your new product is designed and made to the highest standards to ensure excellent performance and easy operation and maintenance. Please read the operation manual thoroughly before using the product, and keep the manual where it is readily available to all users of the product.

This document is intended for medical personnel, dentists and dental assistants.

## Symbols

	Danger symbol.
	Warning symbol.
	Caution symbol.
	Hint symbol.
	Read usage instructions symbol.
	CE Mark (European Community). Products bearing this mark conform to applicable EU Directives.

	Class II medical device
	Applied Part: Type B
	Manufacturer symbol.
	This product is Autoclavable.
	This product can be washed via Thermo Disinfector.
	Do not dispose of with domestic waste.
	Request for assistance

## Identification of warnings

Pay particular attention to the following warnings in order to avoid injury to persons and damage to equipment.

Indicates a risk that can cause death or serious injury if not avoided.

Indicates a risk that can cause serious injury or moderate injury if not avoided.

Indicates a risk that can cause damage to equipment or moderate or slight injury if not avoided.

General explanations, without risk to persons and/or objects.

## Safety precautions

### **WARNING!**

In the event of lesions to soft tissue in the oral cavity, or recent tooth extractions, do not perform treatments using instruments powered by compressed air. Infected substances could enter the lesions and cause infection. There is also a risk of air embolism.

### **CAUTION!**

After treatment, remove the bur or stone before placing the handpiece back in its support. Take care not to drop!

### **CAUTION!**

Electric micromotors produce more power than air turbines and conventional air driven motors. When handpieces are subjected to high torques and speeds, if they have received inadequate maintenance, are damaged or are used for

purposes other than that for which they are designed, they can overheat and cause serious burns to the patient. Stop using the handpiece immediately if you notice unusual functioning, excessive vibrations, overheating or any other indications of a malfunction in the device.

### **WARNING!**

Do not use this medical device as a light probe. Rotating burs or stones can cause oral lesions.

### **CAUTION!**

Clean, service and dry the instrument as instructed before extended periods of disuse.

### **CAUTION!**

Always perform regular functioning checks and maintenance

even after the warranty has expired. The following personnel are authorised to undertake the repair and maintenance of this Ivory product:

- Technicians working in Ivory factories around the world.
- Technicians who have completed Ivory technical training.

Technical assistance must only be performed by qualified Ivory technicians, who can also provide original Ivory spare parts.

 **CAUTION!**

Do not attempt to disassemble the product or tamper with any mechanism. This medical product must be used in the ways illustrated in this manual. The products and systems specified in the manual for cleaning and maintenance must also be used. Users are responsible for regular functioning checks, maintenance and inspection of the product.

 **WARNING!**

Sterilise the handpiece and all accessories before use.

 **CAUTION!**

Dirty or humid cooling air can cause malfunctioning and rapid bearing wear.

Cooling air must be clean, dry and uncontaminated, and must conform to ISO 7494-2 standard.

 **CAUTION!**

Only use clean, undamaged burs to prevent dirt from penetrating into the locking system.

 **DANGER!**

A bur that is loose, unsuitable or improperly seated can work free of the head or break. Before use always confirm that the bur is correctly seated and locked into place.

 **CAUTION!**

Insufficient cooling can lead to overheating of the preparation and even damage the tooth. Make sure that the water flow rate is greater than 50 NL/min.

**⚠ WARNING!**

Do not use the handpiece to move the patient's cheek while the device is functioning!

**⚠ WARNING!**

Do not look at the LED light for a long time when it is illuminated.

**⚠ WARNING!**

Always check the handpiece for damage before using it.

**⚠ WARNING!**

It can be dangerous to use the handpiece without a speed and direction controller.

- The dental unit with which this handpiece is used must be equipped with a speed and direction controller.
- For reasons of legal responsibility for safety, reliability and performance, a statement to this effect must also appear in the documentation provided with the dental unit.

**⚡ DANGER!**

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

**⚠ CAUTION!**

Use of unfiltered, hard water can lead to blockages and premature wear in hoses, fittings and spray heads.

## Product description

Silk Line low speed contra-angle handpiece, push button, internal spray, no optic fibre, ratio 1:1. **REF. SL411C**

Silk Line low speed contra-angle handpiece, push button, internal spray, optic fibre, ratio 1:1. **REF. SL411L**

Silk Line low speed contra-angle handpiece, push button, internal spray, no optic fibre, ratio 2:1. **REF. SL421C**

Silk Line low speed contra-angle handpiece, push button, internal spray, optic fibre, ratio 2:1. **REF. SL421L**

Silk Line high speed contra-angle handpiece, push button, internal spray, no optic fibre, ratio 1:5. **REF. SL415C**

Silk Line high speed contra-angle handpiece, push button, internal spray, optic fibre, ratio 1:5. **REF. SL415L**

Silk Line low speed contra-angle handpiece, push button, external spray, no optic fibre, ratio 20:1. **REF. SL4201C**

## Purpose

This medical device is designed exclusively for the treatment of dental lesions and pathologies in a dental clinic, and conforms to applicable national legal standards. All other uses and modifications to the product are strictly forbidden, and can lead to hazardous situations.

This medical product is intended for the following applications:

- Models SL411C / SL411L / SL421C / SL421L: prophylactic and restorative treatments.
- Models SL415C / SL415L: generic and restorative dental treatments.
- Model SL4201C: implantology.

All uses of this device other than those intended, and the use of accessories and spare parts not authorised by Ivory is strictly forbidden and can even be dangerous.

Failure to conform to the instructions provided herein invalidates the warranty and exonerates the manufacturer from any other claims. This product must only be used by specialist dental personnel, in a dental clinic or laboratory. In conformity to these requirements, this product must only be used by personnel specialising in its intended uses. The following requirements must be respected:

- provisions governing safety in the workplace;
- provisions governing accident prevention;
- the instructions provided in this manual.

### **The user must:**


- use only perfectly functional equipment;
- use the product only for its intended purpose;
- protect himself, patients and third parties against potential hazards;
- avoiding contamination by the product.

## Technical data

Product code	SL411C	SL411L	SL421C	SL421L	SL415C	SL415L	SL4201C
Indications	Prophylaxis	Prophylaxis	Prophylaxis	Prophylaxis	Restorative	Restorative	Implantology
Ratio	1:1	1:1	2:1	2:1	1:5	1:5	20:1
Light intensity up to (lux)	/	40,000	/	40,000	/	40,000	/
Locking mechanism	Button	Button	Button	Button	Button	Button	Button
Type of spray	Internal	Internal	Internal	Internal	Internal	Internal	External (2)
Type of coupling	Intramatic universal	Intramatic universal	Intramatic universal	Intramatic universal	Intramatic universal	Intramatic universal	Intramatic universal
Rotation speed (rpm) (micromotor speed set to 40,000 rpm)	40,000	40,000	20,000	20,000	200,000	200,000	2,000
Torque (Ncm)	/	/	/	/	/	/	≥ 55
Removable head	Yes	Yes	Yes	Yes	No	No	No

<b>Product code</b>	<b>SL411C</b>	<b>SL411L</b>	<b>SL421C</b>	<b>SL421L</b>	<b>SL415C</b>	<b>SL415L</b>	<b>SL4201C</b>
Rotatable head	No	No	No	No	No	No	No
Head diameter (mm)	9.5	9.5	9.5	9.5	9.9	9.9	9.5
Head height (mm)	11.6	11.6	11.6	11.6	14.5	14.5	13.5
Handpiece weight (g)	79	79	79	79	82	82	58
Bur/file diameter (mm)	2.334/2.350	2.334/2.350	2.334/2.350	2.334/2.350	1.59/1.60	1.59/1.60	2.334/2.350
Bur/file length (mm)	21.95	21.95	21.95	21.95	10/25	10/25	21.95
Type of lubrication	Oil	Oil	Oil	Oil	Oil	Oil	Oil
Autoclave resistant	Yes, 135°C	Yes, 135°C	Yes, 135°C	Yes, 135°C	Yes, 135°C	Yes, 135°C	Yes, 135°C
Packing size lxdxh (mm)	160x90x43	160x90x43	160x90x43	160x90x43	160x90x43	160x90x43	160x90x43
Packing weight appr. (g)	250	250	250	250	250	250	250
Certification	CE 0476	CE 0476	CE 0476	CE 0476	CE 0476	CE 0476	CE 0476
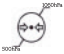
## Transport and storage conditions

	Temperature: from -40 °C to +70 °C (from 40 °F to +158 °F).
	Relative humidity: from 10 % to 80 % non-condensing.
	Atmospheric pressure: from 500 hPa to 1060 hPa (from 7 to 15 psi).
	Protect against humidity.
	Fragile.

## Use

### CAUTION!

Read the safety precautions before starting to use the instrument.

### CAUTION!

At the start of every working day and after the treatment of every patient, the water-conducting systems should be rinsed for at least 2 min. without the instrument being attached.

### CAUTION!


Do not disconnect the handpiece from the micromotor while it is functioning.

### DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

## Fitting and removing the head (only models SL411C, SL411L, SL421C, SL421L)


- 1) Turn the locking ring (1) fully anti-clockwise and push in the head (2) as far as it will go.
- 2) Release the locking ring (1).
- 3) Check that the head is securely held in the handpiece.
- 4) Turn the locking ring (1) fully anti-clockwise to remove the head (2).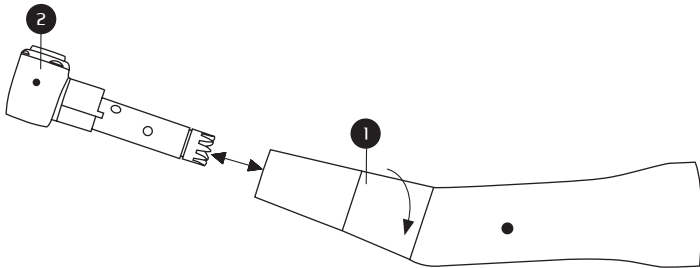

## Fitting and removing the handpiece

**ONLY** use this contra-angle handpiece with type E micromotors (ISO 3964).

- 1) Push the handpiece on to the micromotor until it clicks into place. In the case of handpieces with optic fibre, turn the handpiece until you hear a click, indicating correct alignment.
- 2) Check that the handpiece is securely connected to the micromotor.
- 3) Wait for the micromotor to stop before disconnecting the handpiece from it.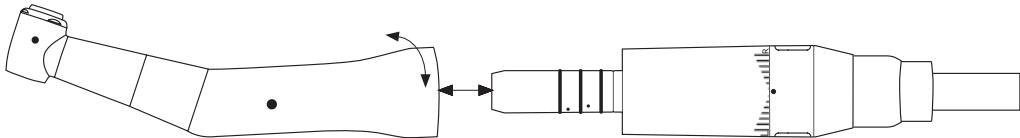

## Fitting and removing the spray nozzle (only model SL4201C)

Contra-angle model SL4201C can be used with three spray delivery methods: with external spray nozzle only, with internal spray nozzle only, or with both internal and external spray nozzles.


### - External spray

1) Connect the spray tube (B) to the handpiece's external spray nozzle (A2).

### Internal spray


- 1) Fit the nozzle support (D) to the handpiece.
- 2) Connect the spray tube (B) to the internal spray nozzle (A1).
- 3) Insert the internal spray nozzle (A1) into the push-button hole and secure it to the support (D).

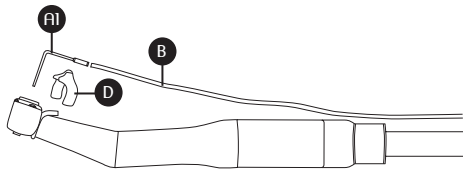## Internal and external spray

- 1) Connect the spray tube (B1) to the handpiece's external spray nozzle (A1).
- 2) Fit the nozzle support (D) to the handpiece.
- 2) Connect the spray tube (B2) to the internal spray nozzle (A2).
- 4) Insert the internal spray nozzle (A1) into the push-button hole and secure it to the support (D).
- 5) Connect the two spray tubes (B1-B2) to the Y union.
- 6) Connect the supply tube (B) to the Y union.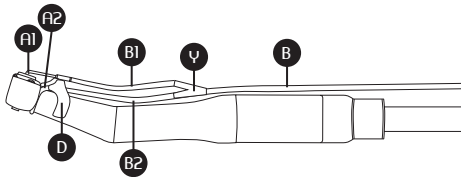

## Fitting and removing the rotary bur

### ⚠ CAUTION!

Never use burs that are bent, deformed, worn, corroded, broken or defective, burs with cracks in the edges or shafts, burs that do not conform to ISO standards, burs that have been tampered with, or burs that do not conform to the specifications given herein. Follow the instructions provided by the manufacturer of the bur you are using. Only use burs that satisfy the following criteria:

- Bur shaft diameter: 2.334/2.350 mm, type 1 according to ISO 1797-1, max. length 21.95 mm, code 4 according to ISO 6360-1 for ratios 1:1, 2:1 and 20:1.
- Bur shaft diameter: 1.59/1.60 mm, type 3 according to ISO 1797-1, max. length from 10 to 25 mm, code 4/5 according to ISO 6360-1 for ratio 1:5.

### ⚡ DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

 **DANGER!**

Only fit the bur in the handpiece when the handpiece is stopped. Never attempt to grip a bur while it is rotating, even when it is nearly stopped.

 **DANGER!**

Never press the locking button on the handpiece while the instrument is in use or has not quite stopped. Doing so can cause the bur to become loose or cause the locking system to overheat and break.

 **DANGER!**

AVOID CONTACT between the patient's cheek and the locking button during use. This could lead to the bur being accidentally released, resulting in burns or lesions to the patient.

 **WARNING!**


Never operate the handpiece without a bur in the spindle.

This could cause the handpiece to overheat, causing damage or burns. Never exceed the maximum speed specified by the bur manufacturer. Keep bur shafts clean at all times; dirty shafts can cause dirt to enter the spindle, leading to the bur slipping during rotation, or even preventing the bur from fitting in the spindle correctly. Never attempt to fit an instrument to a rotating micromotor.


 **DANGER!**

Never apply excessive pressure to burs. They could break, bend or become difficult to extract.


1) Press the locking button (1) and push the bur (2) in as far as it will go. On blue and green contra-angle handpieces, turn the bur until it locks in place.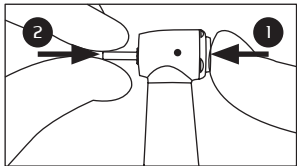

2) Check that the bur (2) is held firmly in place.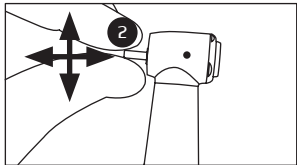

3) Press the locking button (1) to release the bur (2).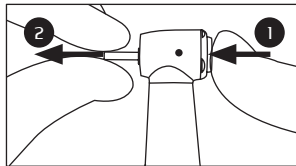

 **DANGER!**

If you notice any unusual noise, vibration, overheating, interruption in the flow of cooling liquid, leakage or changes in the colour of the LED, stop using

 **CAUTION!**

Always check that the cooling system is unobstructed before starting work.

## Hygiene and maintenance

### **WARNING!**

Respect all directives, standards and national legislation governing cleaning, disinfection and sterilisation.

### **DANGER!**

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

### **CAUTION!**

Never immerse the handpiece in disinfectant solutions or ultrasonic baths! Only use chlorine-free disinfectant products that have been certified by officially recognised institutes.


### -External disinfection

Clean and disinfect the handpiece after every treatment to eliminate residues of bodily fluids (e.g. blood, saliva etc.) and prevent them entering the internal parts of the handpiece. On models with a detachable head, remove the head from the handpiece.

- 1) Clean off all dirt and impurities from the handpiece. Do not use metal brushes.
- 2) Clean the handpiece with disinfectant wipes.
- 3) Blow the handpiece dry with a jet of compressed air.

### - Cleaning the spray nozzles

Use the cleaning wire provided to gently remove dirt and deposits from the spray nozzles. Clean and disinfect the cleaning wire.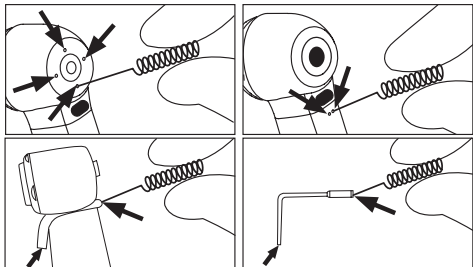


#### **! CAUTION!**

Do not force the brush into the conduit. Damage to the conduit can change the direction of spray with respect to the bur and reduce the effectiveness of cooling.

### - Cleaning the optic fibre glass (if fitted)

Clean the glass of the optic fibre with a pad dampened in alcohol. Remove all residual impurities.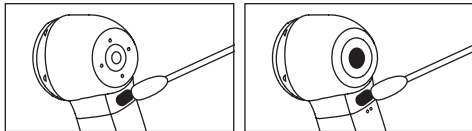

### - Lubrication

#### **! CAUTION!**

Lubrication cycles:


- After every 15 minutes of rotation.
- Before every sterilisation.

#### **! CAUTION!**


Lubricate only with Ivory oil OL101S.

## Procedure for models SL411C, SL411L, SL421C and SL421L


1) Turn the locking ring (1) fully anti-clockwise and remove the head.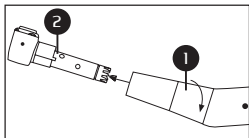

2) Fit the adapter for Ivory heads and contra-angles to the oil spray can. Fit the head (A) in the adapter (B). Hold the head in place. Spray for about 5 seconds.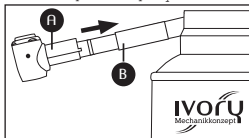



3) Apply the adapter to the back of the handpiece. Hold the handpiece in place and spray for about 5 seconds.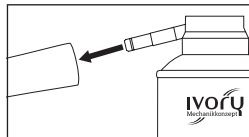

4) Replace the head in the handpiece.


### Procedure for models SL415C, SL415L and SL4201C

1) Fit the universal Ivory turbine handpiece adapter to the oil spray can. Insert the other end of the universal Ivory adapter into the chuck in the handpiece. Hold the handpiece in place. Spray for about 5 seconds.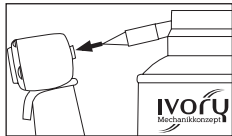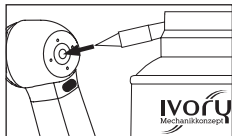

2) Fit the adapter for Ivory heads and contra-angles to the oil spray can. 3) Apply the adapter to the back of the

handpiece. Hold the handpiece in place and spray for about 5 seconds.

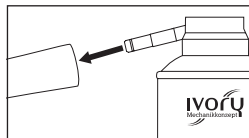#### **! CAUTION!**

Incorrect or inadequate spindle cleaning can cause the spindle to loosen and accidentally release the bur during functioning.

#### **! CAUTION!**

After lubricating, hold the handpiece with the chuck facing down. Operate the handpiece for 30 seconds to expel any excess oil. If you notice any dirt coming out, repeat the entire cleaning and lubrication process.

## -Sterilisation

### Sterilisation in an autoclave conforming to: EN 13060, EN ISO 17665-1

#### CAUTION!

- Follow the instructions of the autoclave manufacturer.
- Clean, disinfect and lubricate the handpiece before sterilisation. The presence of blood residues on the external or internal surfaces can lead to the formation of corrosion.
- Make sure that the handpiece is properly dried after sterilisation.

#### Authorised sterilisation procedures

- 1) Seal the handpiece and its accessories in sterilisation bags conforming to EN 868-5.
- 2) Sterilise the bagged handpiece in an autoclave according to **EN ISO 17665-1** standard and respect the following conditions:
  - Class B steam sterilisation in autoclaves conforming to

**EN 13060** standard. Sterilisation time at least 3.5 minutes at 135 °C.

- 3) Remove the bag from the autoclave.

## -Storage

- Keep the handpiece in the sterilisation bag until it is needed.
- Once sterilised, store the product in a dry and dust-free place, if possible with a low microbial charge.

#### CAUTION!

Respect the validity date for the sterilisation!

## Disposal

-  Sterilise the product and its accessories before disposing of them. Refer the disposal of the product to authorised companies qualified to dispose of special industrial waste.

## Technical assistance – Ivory direct line

HINT!

To request technical assistance, compile the on-line form available at: [www.ivory-dent.com](http://www.ivory-dent.com)

## Declaration of conformity

This medical device conforms to the requirements of European Directives 93/42 CEE (Class IIa) and 2007/47/CE.  
This conformity is expressed by the mark:

CE  
0476

Additional documentation for controller n°0476 can be requested from: [ivory@ivory-dent.com](mailto:ivory@ivory-dent.com)

## Warranty

Ivory guarantees this product against defects of material and manufacture for a period of 24 months from the date of purchase. Ivory cannot be held responsible for defects and consequences caused by or attributable to normal wear, improper use, cleaning or maintenance, failure to follow operating and coupling instructions, lime scale or corrosion, contamination of the air or water supplies, chemical or electrical effects of an exceptional nature or not permitted by the instructions provided by Ivory or other manufacturers. Ivory expressly declines all responsibility for defects or consequences attributable to tampering with the product or modifications made by the customer or unauthorised third parties, or attributable to the use of non-original spare parts. In the event of justified claims, Ivory shall either repair the product or supply spare parts free of charge under the warranty. The warranty excludes all other claims, with particular reference to claims for compensation. In the case of delays, serious or wilful negligence, this exclusion is only valid provided no specific laws are contravened.

The warranty does not apply to lamps, glass and optic fibre conductors, glass or rubber parts, or the colour fastness of plastic materials or all parts subject to normal wear. Claims made under the warranty must be presented by compiling the on-line request "QRRMA" form and by accompanying the product with a copy of the purchase invoice.

The purchase invoice must clearly identify the dealer, date of purchase, code or model of the product and the production or serial number. Please refer also to the articles 17,18 and 19 in the "Terms and payment" section on [www.ivory-dent.com](http://www.ivory-dent.com)

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KaVo® is a registered trademark of Kaltenbach & Voigt GmbH & Co., Germany. Sirona® is a registered trademark of Sirona Dental Systems GmbH, Germany. Bien-Air® is a registered trademark of Bien-Air Dental S.A., Switzerland. NSK® is a registered trademark of NSK NAKANISHI INC., Japan. W&H® and Roto Quick® are registered trademarks of W&H Dentalwerk Bürmoos GmbH, Austria.