# Silk Line

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# English

## Contents

Page	Chapter
24	Information for the user
24	Symbols
25	Identification of warnings
26	Safety precautions
29	Product description
30	Purpose
31	Technical data
33	Transport and storage conditions
33	Use
37	Hygiene and maintenance
40	Disposal
40	Technical Assistance
41	Certification
42	Warranty

## Information for the user

Dear Customer,

Thank you for purchasing this product from Ivory. Your new product is designed and made to the highest standards to ensure excellent performance and easy operation and maintenance. Please read the operation manual thoroughly before using the product, and keep the manual where it is readily available to all users of the product.

This document is intended for medical personnel, dentists and dental assistants.

## Symbols

	Danger symbol.
	Warning symbol.
	Caution symbol.
	Hint symbol.
	Read usage instructions symbol.
	CE Mark (European Community). Products bearing this mark conform to applicable EU Directives.

	Class II medical device
	Applied Part: Type B
	Manufacturer symbol.
	This product is Autoclavable.
	This product can be washed via Thermo Disinfector.
	Do not dispose of with domestic waste.
	Request for assistance

## Identification of warnings

Pay particular attention to the following warnings in order to avoid injury to persons and damage to equipment.

### **DANGER!**

Indicates a risk that can cause death or serious injury if not avoided.

### **WARNING!**

Indicates a risk that can cause serious injury or moderate injury if not avoided.

### **CAUTION!**

Indicates a risk that can cause damage to equipment or moderate or slight injury if not avoided.

### **HINT!**

General explanations, without risk to persons and/or objects.

## Safety precautions

### **WARNING!**

In the event of lesions to soft tissue in the oral cavity, or recent tooth extractions, do not perform treatments using instruments powered by compressed air. Infected substances could enter the lesions and cause infection. There is also a risk of air embolism.

### **CAUTION!**

After treatment, remove the bur or stone before placing the handpiece back in its support. Take care not to drop!

### **CAUTION!**

Electric micromotors produce more power than air turbines and conventional air driven motors. When handpieces are subjected to high torques and speeds, if they have received

inadequate maintenance, are damaged or are used for purposes other than that for which they are designed, they can overheat and cause serious burns to the patient. Stop using the handpiece immediately if you notice unusual functioning, excessive vibrations, overheating or any other indications of a malfunction in the device.

### **WARNING!**

Do not use this medical device as a light probe. Rotating burs or stones can cause oral lesions.

### **CAUTION!**

Clean, service and dry the instrument as instructed before extended periods of disuse.

### **CAUTION!**

Always perform regular functioning checks and maintenance even after the warranty has expired. The following personnel are authorised to undertake the repair and maintenance of this Ivory product:

- Technicians working in Ivory factories around the world.
  - Technicians who have completed Ivory technical training.
- Technical assistance must only be performed by qualified Ivory technicians, who can also provide original Ivory spare parts.

 **CAUTION!**

Do not attempt to disassemble the product or tamper with any mechanism. This medical product must be used in the ways illustrated in this manual. The products and systems specified in the manual for cleaning and maintenance must also be used. Users are responsible for regular functioning checks, maintenance and inspection of the product.

 **WARNING!**

Sterilise the handpiece and all accessories before use.

 **CAUTION!**

Dirty or humid cooling air can cause malfunctioning and rapid bearing wear. Cooling air must be clean, dry and uncontaminated, and must conform to ISO 7494-2 standard.

 **CAUTION!**

Only use clean, undamaged burs to prevent dirt from penetrating into the locking system.

 **DANGER!**

A bur that is loose, unsuitable or improperly seated can work free of the head or break. Before use always confirm that the bur is correctly seated and locked into place.

 **CAUTION!**

Insufficient cooling can lead to overheating of the preparation and even damage the tooth. Make sure that the water flow rate is greater than 50 NL/min.

 **WARNING!**

Do not use the handpiece to move the patient's cheek while the device is functioning!

 **WARNING!**

Do not look at the LED light for a long time when it is illuminated.

 **WARNING!**

Always check the handpiece for damage before using it.

 **WARNING!**

It can be dangerous to use the handpiece without a speed and direction controller.

- The dental unit with which this handpiece is used must be equipped with a speed and direction controller.
- For reasons of legal responsibility for safety, reliability and performance, a statement to this effect must also appear in the documentation provided with the dental unit.

 **DANGER!**

Always wear gloves to avoid the risk of lesions and

infections when checking, cleaning, repairing or extracting teeth.

 **CAUTION!**

Use of unfiltered, hard water can lead to blockages and premature wear in hoses, fittings and spray heads.

## Product description

Silk Line low speed straight handpiece, internal spray, no optic fibre, ratio 1:1. **MOD. SL211C**

Silk Line low speed straight handpiece, internal spray, optic fibre, ratio 1:1. **MOD. SL211L**

## Purpose

This medical device is designed exclusively for the treatment of dental lesions and pathologies in a dental clinic, and conforms to applicable national legal standards. All other uses and modifications to the product are strictly forbidden, and can lead to hazardous situations.

This medical product is intended for the following applications: dental treatments / restorations (implants, grinding, polishing).

All uses of this device other than those intended, and the use of accessories and spare parts not authorised by Ivory is strictly forbidden and can even be dangerous.

Failure to conform to the instructions provided herein invalidates the warranty and exonerates the manufacturer from any other claims. This product must only be used by

specialist dental personnel, in a dental clinic or laboratory. In conformity to these requirements, this product must only be used by personnel specialising in its intended uses. The following requirements must be respected:

- provisions governing safety in the workplace;
- provisions governing accident prevention;
- the instructions provided in this manual.

### **The user must:**

- use only perfectly functional equipment;
- use the product only for its intended purpose;
- protect himself, patients and third parties against potential hazards;
- avoiding contamination by the product.


## Technical data

Product code	SL211C	SL211L
Indications	Dental treatments, restorations	Dental treatments, restorations
Ratio	1:1	1:1
Light intensity up to (lux)	/	40,000
Type of spray	Internal	Internal
Type of coupling	Intramatic Universal	Intramatic Universal
Rotation speed (rpm) (micromotor speed set to 40,000 rpm)	40,000	40,000
Handpiece weight (g)	93	95
Bur diameter (mm)	2.334 / 2.350	2.334 / 2.350

<b>Product code</b>	<b>SL211C</b>	<b>SL211L</b>
Bur length (mm)	21.95 / 44.50	21.95/44.50
Type of lubrication	Oil	Oil
Autoclave resistant	Yes, 135°C	Yes, 135°C
Packing size lxdxh (mm)	160 x 90 x 43	160 x 90 x 43
Packing weight appr. (g)	250	250
Certification	CE 0476	CE 0476
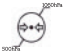
## Transport and storage conditions

	Temperature: from -40 °C to +70 °C (from 40 °F to +158 °F).
	Relative humidity: from 10 % to 80 % non-condensing.
	Atmospheric pressure: from 500 hPa to 1060 hPa (from 7 to 15 psi).
	Protect against humidity.
	Fragile.

## Use

### CAUTION!

Read the safety precautions before starting to use the instrument.

### CAUTION!

At the start of every working day and after the treatment of every patient, the water-conducting systems should be rinsed for at least 2 min. without the instrument being attached.

### CAUTION!

Do not disconnect the handpiece from the micromotor while it is functioning.


### DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

## Fitting and removing the micromotor

**ONLY** use this contra-angle handpiece with type E micromotors (ISO 3964).

- 1) Push the handpiece over the micromotor and turn it until it clicks into place.
- 2) Check that the handpiece is securely connected to the micromotor.
- 3) Wait for the micromotor to stop before disconnecting the handpiece from it.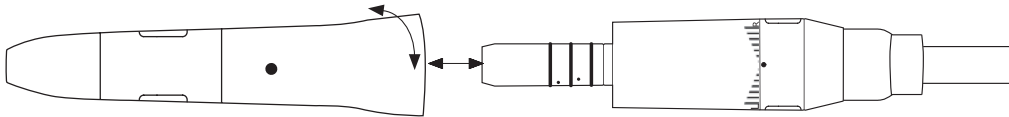

## Fitting and removing the rotary bur

### CAUTION!

Never use burs that are bent, deformed, worn, corroded, broken or defective, burs with cracks in the edges or shafts, burs that do not conform to ISO standards, burs that have been tampered with, or burs that do not conform to the specifications given herein. Follow the instructions provided by the manufacturer of the bur you are using.

Only use burs that satisfy the following criteria: shaft diameter 2.334 / 2.350 mm type 2 according to ISO 1797-1, max. bur length 44.5 mm code 4 according to ISO 6360/1.

### DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

### DANGER!

Only fit the bur in the handpiece when the handpiece is

stopped. Never attempt to grip a bur while it is rotating, even when it is nearly stopped.

### DANGER!

Never press the locking button on the handpiece while the instrument is in use or has not quite stopped. Doing so can cause the bur to become loose or cause the locking system to overheat and break.

### WARNING!

Never operate the handpiece without a bur in the spindle. This could cause the handpiece to overheat, causing damage or burns. Never exceed the maximum speed specified by the bur manufacturer. Keep bur shafts clean at all times; dirty shafts can cause dirt to enter the spindle, leading to the bur slipping during rotation, or even

preventing the bur from fitting in the spindle correctly. Never attempt to fit an instrument to a rotating micromotor.

**⚡ DANGER!**

Never apply excessive pressure to burs. They could break, bend or become difficult to extract.

**⚡ DANGER!**

If you notice any unusual noise, vibration, overheating, interruption in the flow of cooling liquid, leakage or changes in the colour of the LED, stop using

**! CAUTION!**


Always check that the cooling system is unobstructed before starting work.

1) Turn the locking ring (1) anti-clockwise until it clicks, and push in the bur (2) as far as it will go.

2) Turn the locking ring (1) clockwise until it clicks.

3) Check that the rotary bur is securely held in the handpiece.

4) Turn the locking ring (1) anti-clockwise until it clicks, and remove the rotary bur (2).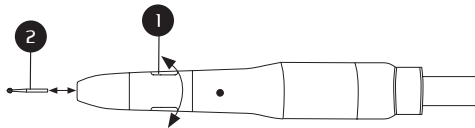

## Hygiene and maintenance

### **WARNING!**

Respect all directives, standards and national legislation governing cleaning, disinfection and sterilisation.

### **DANGER!**

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

### **CAUTION!**

Never immerse the handpiece in disinfectant solutions or ultrasonic baths! Only use chlorine-free disinfectant products that have been certified by officially recognised institutes.

### -External disinfection


Clean and disinfect the handpiece after every treatment

to eliminate residues of bodily fluids (e.g. blood, saliva etc.) and prevent them entering the internal parts of the handpiece.

- 1) Clean off all dirt and impurities from the handpiece. Do not use metal brushes.
- 2) Clean the handpiece with disinfectant wipes.
- 3) Blow the handpiece dry with a jet of compressed air.

### - Cleaning the spray nozzles

Use the cleaning wire provided to gently remove dirt and deposits from the spray nozzles. Clean and disinfect the cleaning wire.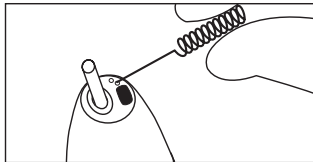


**! CAUTION!**

Do not force the brush into the conduit. Damage to the conduit can change the direction of spray with respect to the bur and reduce the effectiveness of cooling.

**- Cleaning the optic fibre glass (if fitted)**

Clean the glass of the optic fibre with a pad damped in alcohol. Remove all residual impurities.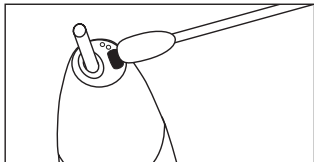

**- Lubrication**

**! CAUTION!**


Lubrication cycles:

- After every 15 minutes of rotation.
- Before every sterilisation.

**! CAUTION!**


Lubricate only with Ivory oil OL101S.

1) Fit the universal Ivory turbine handpiece adapter to the oil spray can. Insert the other end of the universal Ivory adapter into the chuck in the handpiece. Hold the handpiece in place. Spray for about 5 seconds.

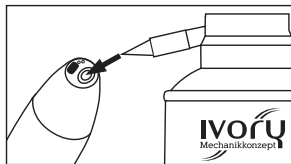2) Fit the adapter for Ivory heads and contra-angles to the oil spray can. 3) Apply the adapter to the back of the handpiece. Hold the handpiece in place and spray for about 5 seconds.

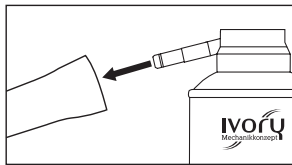**! CAUTION!**

Incorrect or inadequate spindle cleaning can cause the spindle to loosen and accidentally release the bur during functioning.

**! CAUTION!**

After lubricating, hold the handpiece with the chuck facing down. Operate the handpiece for 30 seconds to

expel any excess oil. If you notice any dirt coming out, repeat the entire cleaning and lubrication process.

**-Sterilisation**

**Sterilisation in an autoclave conforming to: EN 13060, EN ISO 17665-1**

**! CAUTION!**

- Follow the instructions of the autoclave manufacturer.
- Clean, disinfect and lubricate the handpiece before sterilisation. The presence of blood residues on the external or internal surfaces can lead to the formation of corrosion.
- Make sure that the handpiece is properly dried after sterilisation.

**Authorised sterilisation procedures**

- 1) Seal the handpiece and its accessories in sterilisation bags conforming to EN 868-5.
- 2) Sterilise the bagged handpiece in an autoclave

according to **EN ISO 17665-1** standard and respect the following conditions:

- Class B steam sterilisation in autoclaves conforming to **EN 13060** standard. Sterilisation time at least 3.5 minutes at 135 °C.

3) Remove the bag from the autoclave.

### **-Storage**

- Keep the handpiece in the sterilisation bag until it is needed.

- Once sterilised, store the product in a dry and dust-free place, if possible with a low microbial charge.

### **CAUTION!**

Respect the validity date for the sterilisation!

## Disposal

Sterilise the product and its accessories before disposing of them. Refer the disposal of the product to authorised companies qualified to dispose of special industrial waste.

## Technical assistance – Ivory direct line

### **HINT!**

To request technical assistance, compile the on-line form available at: [www.ivory-dent.com](http://www.ivory-dent.com)

## Declaration of conformity

This medical device conforms to the requirements of European Directives 93/42 CEE (Class IIa) and 2007/47/CE.  
This conformity is expressed by the mark:

Additional documentation for controller n°0476 can be requested from: [\*ivory@ivory-dent.com\*](mailto:ivory@ivory-dent.com)

## Warranty

Ivory guarantees this product against defects of material and manufacture for a period of 24 months from the date of purchase. Ivory cannot be held responsible for defects and consequences caused by or attributable to normal wear, improper use, cleaning or maintenance, failure to follow operating and coupling instructions, lime scale or corrosion, contamination of the air or water supplies, chemical or electrical effects of an exceptional nature or not permitted by the instructions provided by Ivory or other manufacturers. Ivory expressly declines all responsibility for defects or consequences attributable to tampering with the product or modifications made by the customer or unauthorised third parties, or attributable to the use of non-original spare parts. In the event of justified claims, Ivory shall either repair the product or supply spare parts free of charge under the warranty. The warranty excludes all other claims, with particular reference to claims for compensation. In the case of delays, serious or wilful negligence, this exclusion is only valid provided no specific laws are contravened.

The warranty does not apply to lamps, glass and optic fibre conductors, glass or rubber parts, or the colour fastness of plastic materials or all parts subject to normal wear. Claims made under the warranty must be presented by compiling the on-line request "QRRMA" form and by accompanying the product with a copy of the purchase invoice.

The purchase invoice must clearly identify the dealer, date of purchase, code or model of the product and the production or serial number. Please refer also to the articles 17,18 and 19 in the "Terms and payment" section on [www.ivory-dent.com](http://www.ivory-dent.com)

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KaVo® is a registered trademark of Kaltenbach & Voigt GmbH & Co., Germany. Sirona® is a registered trademark of Sirona Dental Systems GmbH, Germany. Bien-Air® is a registered trademark of Bien-Air Dental S.A., Switzerland. NSK® is a registered trademark of NSK NAKANISHI INC., Japan. W&H® and Roto Quick® are registered trademarks of W&H Dentalwerk Bürmoos GmbH, Austria.