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English

Contents

Page	Chapter
39	Information for the user
40	Symbols
42	Identification of warnings
43	Safety precautions
48	Product description
49	Purpose
51	Technical data
52	Transport and storage conditions
53	Use
60	Hygiene and maintenance
69	Disposal
69	Technical Assistance
70	Certification
70	Warranty

Information for the user

Dear Customer,

Thank you for purchasing this product from Ivory. Your new product is designed and made to the highest standards to ensure excellent performance and easy operation and maintenance. Please read the operation manual thoroughly before using the product, and keep the manual where it is readily available to all users of the product.

This document is intended for medical personnel, dentists and dental assistants.

Symbols

	Danger symbol.
	Warning symbol.
	Caution symbol.
	Hint symbol.
	Read usage instructions symbol.

	<p>Manufacturer symbol.</p>
	<p>Date of manufacture</p>
	<p>This product is Autoclavable.</p>
	<p>This product can be washed via Thermo Disinfector.</p>
	<p>Do not dispose of with domestic waste.</p>
	<p>Request for assistance</p>

Identification of warnings

Pay particular attention to the following warnings in order to avoid injury to persons and damage to equipment.

Indicates a risk that can cause death or serious injury if not avoided.

Indicates a risk that can cause serious injury or moderate injury if not avoided.

Indicates a risk that can cause damage to equipment or moderate or slight injury if not avoided.

General explanations, without risk to persons and/or objects.

Safety precautions

WARNING!

In the event of lesions to soft tissue in the oral cavity, or recent tooth extractions, do not perform treatments using instruments powered by compressed air. Infected substances could enter the lesions and cause infection. There is also a risk of air embolism.

CAUTION!

After treatment, remove the bur or stone before placing the handpiece back in its support. Take care not to drop!

CAUTION!

Electric micromotors produce more power than air turbines and conventional air driven motors. When handpieces are subjected to high torques and speeds, if they have received inadequate maintenance, are damaged or

are used for purposes other than that for which they are designed, they can overheat and cause serious burns to the patient. Stop using the handpiece immediately if you notice unusual functioning, excessive vibrations, overheating or any other indications of a malfunction in the device.

 **CAUTION!**

Clean, service and dry the instrument as instructed before extended periods of disuse.

 **CAUTION!**

Always perform regular functioning checks and maintenance even after the warranty has expired. The following personnel are authorised to undertake the repair and maintenance of this Ivory product:

- Technicians working in Ivory factories around the world.
- Technicians who have completed Ivory technical training.

Technical assistance must only be performed by qualified Ivory technicians, who can also provide original Ivory spare parts.

! CAUTION!

Do not attempt to disassemble the product or tamper with any mechanism. This medical product must be used in the ways illustrated in this manual. The products and systems specified in the manual for cleaning and maintenance must also be used. Users are responsible for regular functioning checks, maintenance and inspection of the product.

! WARNING!

Sterilise the head and the accessories before use.

! CAUTION!

Dirty or humid cooling air can cause malfunctioning and rapid bearing wear. Cooling air must be clean, dry and uncontaminated, and must conform to UNI EN ISO 7494-2:2015 standard.

! CAUTION!

Only use clean, undamaged burs to prevent dirt from penetrating into the locking system.

 **DANGER!**

A bur that is loose, unsuitable or improperly seated can work free of the head or break. Before use always confirm that the bur is correctly seated and locked into place.

 **CAUTION!**

Insufficient cooling can lead to overheating of the preparation and even damage the tooth. Make sure that the water flow rate is greater than 50 NL/min.

 **WARNING!**

Do not use the handpiece to move the patient's cheek while the device is functioning!

 **WARNING!**

Always check the handpiece for damage before using it.

 **WARNING!**

It can be dangerous to use the handpiece without a speed and direction controller.

- The dental unit with which this handpiece is used must be equipped with a speed and direction controller.
- For reasons of legal responsibility for safety, reliability and performance, a statement to this effect must also appear in the documentation provided with the dental unit.

 **DANGER!**

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

 **CAUTION!**

Use of unfiltered, hard water can lead to blockages and premature wear in hoses, fittings and spray heads.

Product description

Head for contra-angle handpiece, ratio 3:1.

2.35 mm surgical drill, spray duct inside the drill

REF. SL631C

Dental Head for low speed contra-angles with ratio from 4:1 to 9:1 and INTRA KAVO head Type

Head for contra-angle handpiece, ratio 3:1.

2.35 mm surgical drill with hexagonal socket, spray duct inside and outside the drill.

REF. SL620X

Dental Head for low speed contra-angles with ratio 7,4 and INTRA KAVO head Type

Purpose

This replacement part is suitable for medical devices that comply with the corresponding provisions of EN ISO 14457 and intended exclusively for the treatment of dental conditions and dental injuries. Changes to intended use or the product that could lead to hazardous situations are prohibited.

The product is intended for the following applications: endodontic procedures and cavity preparation.

DANGER!

All uses of this device other than those intended, and the use of accessories and spare parts not authorised by Ivory is strictly forbidden and can even be dangerous.

Failure to conform to the instructions provided herein invalidates the warranty and exonerates the manufacturer from any other claims. The medical device housing this replacement part must only be used by

dentists, medical surgeons licensed to practice dentistry in compliance with the following:

- provisions governing safety in the workplace;
- provisions governing accident prevention;
- the instructions provided in this manual.


The user must:

- use only perfectly functional equipment;
- use the product only for its intended purpose;
- protect himself, patients and third parties against potential hazards;
- avoiding contamination by the product.

Technical data

Product code	SL631C	SL620X
Indications	Surgery	Surgery
Max speed RPM	10000	10000
Ratio	3:1	3:1
Spray pipe	Inside the drill	Inside/outside the drill
Type of movement	Standard	Standard
Locking mechanism	Button	Button
Head diameter (mm)	9,80	9,80
Head height (mm)	10,00	10,00
Bur diameter (mm)	2,334/2,350	2,334/2,350 (Hexagonal drill)
Type of lubrication	Oil	Oil
Autoclave resistant	Yes 135°C	Yes 135°C
Packing size (mm)	105 x 75 x 25	105 x 75 x 25
Packing weight (g)	120	120

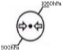Transport and storage conditions

	Temperature: from -40 °C to +70 °C (from 40 °F to +158 °F).
	Relative humidity: from 10 % to 80 % non-condensing.
	Atmospheric pressure: from 500 hPa to 1060 hPa (from 7 to 15 psi).
	Protect against humidity.
	Fragile.

Use

 **CAUTION!**

Read the safety precautions before starting to use the instrument.

 **CAUTION!**

At the start of every working day and after the treatment of every patient, the water-conducting systems should be rinsed for at least 2 min. without the instrument being attached.

 **CAUTION!**


Do not disconnect the handpiece from the micromotor while it is functioning.

 **DANGER!**

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

Fitting and removing the head

- 1) Turn the locking ring (1) fully anti-clockwise and push in the head (2) as far as it will go.
- 2) Release the locking ring (1).
- 3) Check that the head is securely held in the handpiece.
- 4) Turn the locking ring (1) fully anti-clockwise to remove the head (2).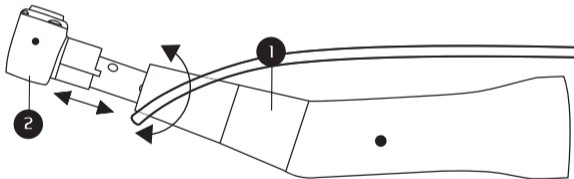

Fitting and removing the rotary bur

CAUTION!

Never use burs that are bent, deformed, worn, corroded, broken or defective, burs with cracks in the edges or shafts, burs that do not conform to ISO standards, burs that have been tampered with, or burs that do not conform to the specifications given herein. Follow the instructions provided by the manufacturer of the bur you are using. Use only surgical drills with internal coolant that comply with EN ISO 1797:2017 and meet the following criteria: drill shank diameter 2.334/2.350 mm, type 1, max. length 45 mm, max. cutting tool diameter 10 mm.

Surgical drills with a hexagonal socket in the shaft area can be used on the SL620X head.

DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

DANGER!

Only fit the bur in the handpiece when the handpiece is stopped.

Never attempt to grip a bur while it is rotating, even when it is nearly stopped.

 **DANGER!**

Never operate the head lock button during use or when stopping. Doing so can cause the bur to become loose or cause the locking system to overheat and break.

 **DANGER!**

AVOID CONTACT between the patient's cheek and the locking button during use. This could lead to the bur being accidentally released, resulting in burns or lesions to the patient.


 **WARNING!**

Do not rotate the head without a drill inserted in the spindle. This could cause the handpiece to overheat, causing damage or burns. Never exceed the maximum speed specified by the bur manufacturer. Keep bur shafts clean at all times; dirty shafts can cause dirt to enter the spindle, leading to the bur slipping during rotation, or even preventing


the bur from fitting in the spindle correctly. Never attempt to fit an instrument to a rotating micromotor.

 **DANGER!**


Never apply excessive pressure to burs. They could break, bend or become difficult to extract.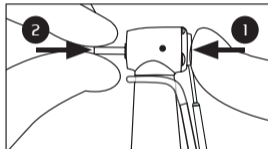

1) Press the locking button (1) and push and turn the bur (2) in as far as it will go.

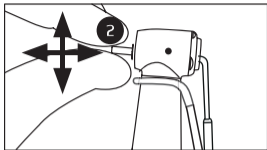
2) Check that the bur (2) is held firmly in place.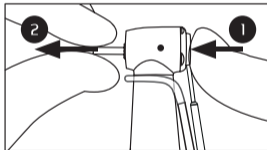

3) Press the locking button (1) to release the bur (2).

Internal spray

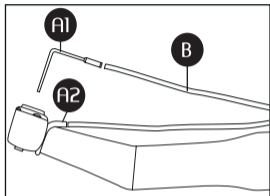1) Connect the watering hose (B) to the inner spray nozzle (A1) and/or the outer spray nozzle A2 (only for SL620X)

2) Insert the internal spray nozzle (A1) into the push-button hole.

DANGER!

If you notice any unusual noise, vibration, overheating, interruption in the flow of cooling liquid, leakage, stop using

CAUTION!

Always check that the cooling system is unobstructed before starting work.

Hygiene and maintenance

WARNING!

Respect all directives, standards and national legislation governing cleaning, disinfection and sterilisation.

DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

CAUTION!

Do not immerse the head in disinfectant solution or ultrasound! Only use chlorine-free disinfectant products that have been certified by officially recognised institutes.

-External disinfection

Remove the head from the handpiece, clean and disinfect the head after each treatment to remove any liquids (e.g. blood, saliva, etc.) and prevent them from depositing on the external parts.

CAUTION!

The saline solution is extremely aggressive and can rust steels in no time.

Thoroughly clean the surfaces using a clean cloth soaked in a suitable substance. Also use a medium hard toothbrush and brush under running hot tap water $30^{\circ}\text{C} \pm 5^{\circ}\text{C}$ ($86^{\circ}\text{F} \pm 10^{\circ}\text{F}$).

Manual internal cleaning

Use a suitable spray and follow the corresponding instructions and spray the substance inside the head. Immediately after internal cleaning, carry out the disinfection process or the drying process.

- Automatic cleaning

Use an ISO 15883 compliant washer-disinfector that uses an alkaline cleaning agent with a maximum pH of up to 10.

- Disinfection

External manual: disinfect using a clean cloth soaked in a suitable substance (e.g. O-phenylphenol or ethyl alcohol). Products not recommended: disinfectants based on benzalkonium chloride, acetone or glutaraldehyde.

Internal manual: Using a suitable spray, follow the corresponding instructions and spray the substance inside the head. Immediately after internal disinfection, perform the oil lubrication process and / or the sterilisation process.

Automated: Use an ISO 15883 compliant washer-disinfector that uses an alkaline cleaning agent with a maximum pH of up to 10.

- Drying

Manual: in general use dry and clean air compliant with ISO 7494-2.

Automated: the drying phase is normally part of the cleaning programme of the washer-disinfector.

- Lubrication


CAUTION!

Lubrication cycles:


- After every 15 minutes of rotation.
- Before every sterilisation.

CAUTION!


Lubricate only with Ivory oil OL101S.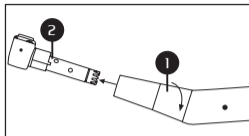

1) Turn the locking ring (1) fully anti-clockwise and remove the head.

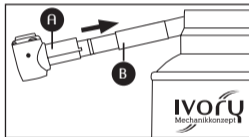
2) Fit the adapter for Ivory heads and contra-angles to the oil spray can. Fit the head (A) in the adapter (B). Hold the head firmly. Spray for about 5 seconds.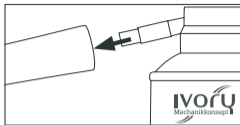


3) Apply the adapter to the back of the handpiece. Hold the handpiece in place and spray for about 5 seconds.

4) Replace the head in the handpiece.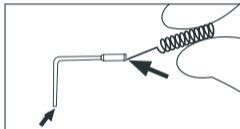

Cleaning of spray nozzles

Use the cleaning wire supplied to carefully remove impurities and internal deposits. Clean and disinfect the cleaning wire before storing it.

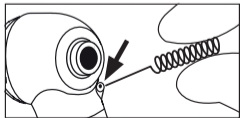! CAUTION!

Incorrect cleaning could loosen the clamping and cause the drill to accidentally detach during operation.

! CAUTION!

After each lubrication, align the head with the drill side downwards. Operate the head for 30 seconds to remove excess oil. If you notice any dirt coming out, repeat the entire cleaning and lubrication process.

-Sterilisation

Sterilisation in an autoclave conforming to: EN 13060, EN ISO 17665-1

! CAUTION!

- Follow the instructions of the autoclave manufacturer.
- Clean, disinfect and lubricate the handpiece before sterilisation. The presence of blood residues on the external or internal surfaces can lead

to the formation of corrosion.

- Make sure that the handpiece is properly dried after sterilisation.

Authorised sterilisation procedures

1) Seal the head and the accessories in bags in accordance with UNI EN 868:2009 standards, UNI EN ISO 11607:2009 "Packaging for terminally sterilised medical devices".

2) Sterilise the bagged handpiece in an autoclave according to **EN ISO 17665-1** standard and respect the following conditions:

- Class B steam sterilisation in autoclaves conforming to **EN 13060** standard.

 The Ivory medical device can withstand temperatures up to 138°C.

Sterilisation time in an autoclave with 3-time pre-vacuum - at least 3 minutes at 134 °C.

3) Remove the bag from the autoclave.

-Storage

- Store the head in the sterile bag until used.
- Once sterilised, store the product in a dry and dust-free place, if possible with a low microbial charge. The processed device must be stored protected from dust and microbiotic contamination. Storage temperature $-10 \div 70^{\circ}\text{C}$. Storage humidity $10 \div 90\%$.

ATTENTION!

Observe the validity date of the sterilisation carried out!

The head is supplied "non-sterile". The head can be sterilised at least 1000 times.

Disposal

Sterilise the product and its accessories before disposing of them.
To dispose of the product, contact Ivory srl or specialised companies authorised to dispose of special industrial waste.

Technical assistance – Ivory direct line

HINT! To request technical assistance, compile the on-line form available at: www.ivory-dent.com

Declaration of conformity

All spare parts are manufactured to have certain specifications and to meet the requirements of the standards: EN ISO 13485 (medical device quality system) - EN ISO 10993-1 (biological evaluation) - EN ISO 17665-1 (sterilisation of medical devices).

Warranty

Ivory guarantees this product against defects of material and manufacture for a period of 24 months from the date of purchase.

Ivory cannot be held responsible for defects and consequences caused by or attributable to normal wear, improper use, cleaning or maintenance, failure to follow operating and coupling instructions, lime scale or corrosion,

contamination of the air or water supplies, chemical or electrical effects of an exceptional nature or not permitted by the instructions provided by Ivory or other manufacturers. Ivory expressly declines all responsibility for defects or consequences attributable to tampering with the product or modifications made by the customer or unauthorised third parties, or attributable to the use of non-original spare parts. In the event of justified claims, Ivory shall either repair the product or supply spare parts free of charge under the warranty. The warranty excludes all other claims, with particular reference to claims for compensation. In the case of delays, serious or wilful negligence, this exclusion is only valid provided no specific laws are contravened.

The warranty does not apply to lamps, glass and optic fibre conductors, glass or rubber parts, or the colour fastness of plastic materials or all parts subject to normal wear. Claims made under the warranty must be presented by compiling the on-line request "QRRMA" form and by accompanying the product with a copy of the purchase invoice.

The purchase invoice must clearly identify the dealer, date of purchase, code or model of the product and the production or serial number. Please refer also to the articles 17,18 and 19 in the "Terms and payment" section on www.ivory-dent.com

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KaVo® is a registered trademark of Kaltenbach & Voigt GmbH & Co., Germany. Sirona® is a registered trademark of Sirona Dental Systems GmbH, Germany. Bien-Air® is a registered trademark of Bien-Air Dental S.A., Switzerland. NSK® is a registered trademark of NSK NAKANISHI INC., Japan. W&H® and Roto Quick® are registered trademarks of W&H Dentalwerk Bürmoos GmbH, Austria.