# Prophy Silk

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# English

## Contents

Page	Chapter
25	Information for the user
25	Symbols
26	Identification of warnings
27	Safety precautions
30	Product description
31	Purpose
32	Technical data
34	Transport and storage conditions
34	Use
39	Hygiene and maintenance
42	Disposal
42	Technical Assistance
42	Certification
43	Warranty

## Information for the user

Dear Customer,

Thank you for purchasing this product from Ivory. Your new product is designed and made to the highest standards to ensure excellent performance and easy operation and maintenance. Please read the operation manual thoroughly before using the product, and keep the manual where it is readily available to all users of the product.

This document is intended for medical personnel, dentists and dental assistants.

## Simboli

	Danger symbol.
	Warning symbol.
	Caution symbol.
	Hint symbol.
	Read usage instructions symbol.
	CE Mark (European Community). Products bearing this mark conform to applicable EU Directives.

	Manufacturer symbol.
	This product is Autoclavable.
	This product can be washed via Thermo Disinfector.
	Do not dispose of with domestic waste.
	Request for assistance

## Identification of warnings

Pay particular attention to the following warnings in order to avoid injury to persons and damage to equipment.

### **DANGER!**

Indicates a risk that can cause death or serious injury if not avoided.

### **WARNING!**

Indicates a risk that can cause serious injury or moderate injury if not avoided.

### **CAUTION!**

Indicates a risk that can cause damage to equipment or moderate or slight injury if not avoided.

### **HINT!**

General explanations, without risk to persons and/or objects.

## Safety precautions

### **WARNING!**

In the event of lesions to soft tissue in the oral cavity, or recent tooth extractions, do not perform treatments using instruments powered by compressed air. Infected substances could enter the lesions and cause infection. There is also a risk of air embolism.

### **WARNING!**

After treatment, place the device correctly, without bicarbonate, in its holder. Do not drop it.

### **WARNING!**

Stop using the handpiece immediately if you notice unusual functioning, excessive vibrations, overheating or any other indications of a malfunction in the device.

### **WARNING!**

Do not use this medical device as a light probe. Rotating burs or stones can cause oral lesions.

### **CAUTION!**

Clean, service and dry the instrument as instructed before extended periods of disuse.

### **CAUTION!**

Always perform regular functioning checks and maintenance even after the warranty has expired. The following personnel are authorised to undertake the repair and maintenance of this Ivory product:

- Technicians working in Ivory factories around the world.
  - Technicians who have completed Ivory technical training.
- Technical assistance must only be performed by qualified Ivory technicians, who can also provide original Ivory spare parts.

**! CAUTION!**

Do not attempt to disassemble the product or tamper with any mechanism. This medical product must be used in the ways illustrated in this manual. The products and systems specified in the manual for cleaning and maintenance must also be used. Users are responsible for regular functioning checks, maintenance and inspection of the product.

**! WARNING!**

Sterilise the handpiece and all accessories before use.

**! CAUTION!**

Dirty or humid cooling air can cause malfunctioning and rapid bearing wear. Cooling air must be clean, dry and uncontaminated, and must conform to ISO 7494-2 standard

**! WARNING!**

Use only bicarbonate for dental use.

**! WARNING!**

Always check the handpiece for damage before using it.

**! CAUTION!**

Check the supply pressure before using the handpiece. To do so, fit a pressure gauge between the dental cord and the coupling. Check the following pressures:

- Operating air pressure: 2.8 bar (40 psi)
- Water: from 0.8 to 2.5 (12 to 36 psi)

Do not exceed the above pressures.

**! WARNING!**

Check that all the O-rings on the coupling are correctly seated and undamaged before using the handpiece.

 **DANGER!**

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

 **CAUTION!**

Use of unfiltered, hard water can lead to blockages and premature wear in hoses, fittings and spray heads.

 **WARNING!**

Make sure that the dispersed cleaning powder does not penetrate into the mucous membranes of the patient's oral cavity (eyes, nose, etc.). The face must also be protected with a cloth or goggles so that cleaning powder does not get into the eyes.

If the patient wears contact lenses, these must be removed before proceeding with the cleaning operation.

 **WARNING!**

If deposits of chemical residues, oil or other substances form on the cap of the powder container, it is recommended to remove them immediately. If this is not done, the cap could be damaged.

 **DANGER!**

Do not use the device on patients:

- with restricted sodium intake
- with severe ulcers in the digestive tract
- with hepatic impairment
- with cardiac or pulmonary dysfunction
- with hyperemia, haemorrhage or inflammation in the oral cavity
- with damage or abnormality in the oral cavity
- with a tendency to develop inflammation in the mucous membranes of the oral cavity
- suffering from allergies
- wearing contact lenses.

## Product description

Whitening Propphy  
handpiece with  
Kavo® compatible  
connection.

**REF. P900K**

Whitening Propphy  
handpiece with  
NSK® compatible  
connection.

**REF. P900N**

Whitening Propphy  
handpiece with  
Sirona® compatible  
connection.

**REF. P900S**

Whitening Propphy  
handpiece with  
BienAir® compatible  
connection.

**REF. P900B**

Whitening Propphy  
handpiece with  
W&H® compatible  
connection.

**REF. P900W**

## Intended use

This medical device, conforming to the corresponding national legal provisions, is intended exclusively for the treatment of dental conditions and diseases in the dental field. Changes in the intended use or alterations to the product that could lead to dangerous situations are not permitted.

The medical product is intended for the following applications:

- cleaning and whitening the surfaces of teeth and restoration.

All uses of this device other than those intended, and the use of accessories and spare parts not authorised by Ivory is strictly forbidden and can even be dangerous. Failure to conform to the instructions provided herein

invalidates the warranty and exonerates the manufacturer from any other claims. This product must only be used by specialist dental personnel, in a dental clinic or laboratory. In conformity to these requirements, this product must only be used by personnel specialising in its intended uses. The following requirements must be respected:

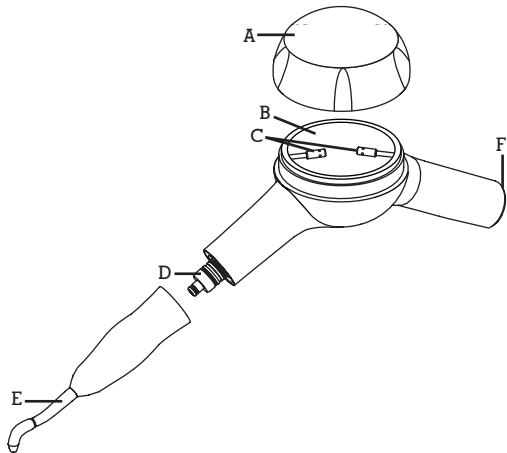
- provisions governing safety in the workplace;
- provisions governing accident prevention;
- the instructions provided in this manual.

### **The user must:**

- use only perfectly functional equipment;
- use the product only for its intended purpose;
- protect himself, patients and third parties against potential hazards;
- avoiding contamination by the product.


## Technical data

Product code	Prophy Silk
Operating pressure	2,8 (40 psi)
Air consumption	10 - 13 NI/min
Water pressure	0,8 - 2,5 bar (12 - 36 psi)
Spray water flow rate	circa 35 - 80 cm <sup>3</sup>
Spray air pressure	1,5 - 2,5 bar (22 - 36 psi)
Autoclavable	Tip only, 135°C
Package size wxdxh (mm)	190 x 125 x 75
Packag weight approx. (g)	500
Certification	CE

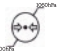- A. Powder container cap
- B. Powder container
- C. Air nozzles
- D. Tip coupling
- E. Tip autoclavable at 135°C
- F. Cord connection

## Transport and storage conditions

	Temperature: from -40 °C to +70 °C (from 40 °F to +158 °F).
	Relative humidity: from 10 % to 80 % non-condensing.
	Atmospheric pressure: from 500 hPa to 1060 hPa (from 7 to 15 psi).
	Protect against humidity.
	Fragile.

## Use

### CAUTION!

Read the safety precautions before starting to use the instrument.

### CAUTION!

At the start of every working day and after the treatment of every patient, the water-conducting systems should be rinsed for at least 2 min. without the instrument being attached.

### CAUTION!

Do not disconnect the handpiece from the coupling while it is functioning.

### DANGER!

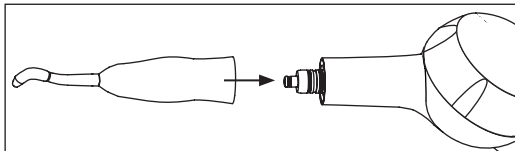
Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

**! WARNING!**

Before reassembling the handpiece, it is recommended to use air to dry all the various parts in order eliminate any trace of dust or water from the device.

**Tip connection**


1) Connect the tip to the handpiece until you hear a click.  
Then gently pull the tip to make sure that it is securely in place


## Insertion and removal of the handpiece (Kavo<sup>®</sup>, Sirona<sup>®</sup>, Nsk<sup>®</sup>, BienAir<sup>®</sup>, W&H<sup>®</sup> couplings)

- 1) Connect the handpiece to the coupling until you hear a click.
- 2) Holding the quick coupling and the cord ring, remove the handpiece by turning it slightly.  
Do not pull on the power cord and do not grip it.

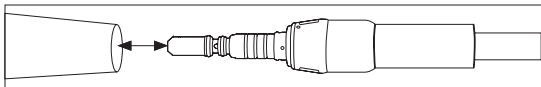
### - KaVo<sup>®</sup> coupling


### - Sirona<sup>®</sup> coupling

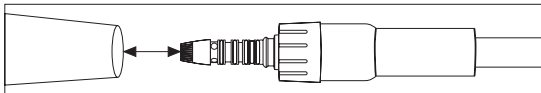
- Nsk® coupling

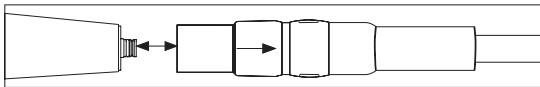- BienAir® coupling

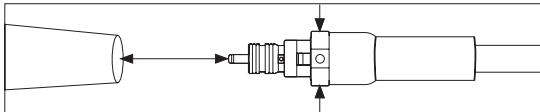- W&H® coupling
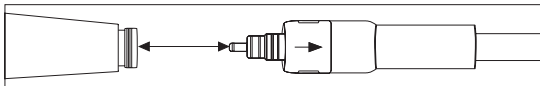
## Filling the powder container

### **! WARNING!**

Do not reuse any leftover powder or any powder that has been left in the bag for a few days. It may be damp and block correct passage of the powder.

### **! WARNING!**

Only use cleaning powder specifically produced for dental use. Powder granules from other brands may actually have different dimensions and may cause a blockage inside the device, thus compromising the effectiveness of cleaning.

- 1- Unscrew the powder container cap by turning it anti-clockwise.
- 2- Before filling the container, shake the powder well in the refill bag.
- 3- Open a bag of cleaning powder and pour the entire contents into the container.
- 4- Fill the powder container up to the mark
- 5- Securely close the container cap by turning it clockwise.

### **⚡ DANGER!**

If you notice unusual noises or changes in normal operation, do not use the handpiece and contact the authorised Ivory distributor.

## Hygiene and maintenance

### WARNING!

Respect all directives, standards and national legislation governing cleaning, disinfection and sterilisation.

### DANGER!

Always wear gloves to avoid the risk of lesions and infections when checking, cleaning, repairing or extracting teeth.

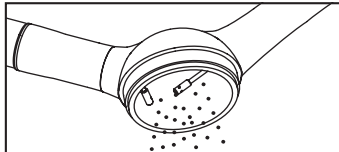
### WARNING!

Do not immerse the handpiece in a disinfectant solution or use ultrasound devices to clean it! Use only chlorine-free disinfectants, certified by officially recognised institutes.


#### **- External disinfection**

Clean and disinfect the handpiece after each treatment to eliminate any liquids (e.g. blood, saliva, etc.) and to prevent them from depositing on internal parts.


- 1) Remove any dirt and impurities from the product. Do not use a wire brush.
- 2) Open the powder container cap and remove any residue.

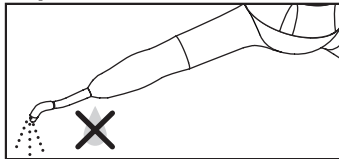- 3) When the container is empty, blow in air to remove any powder and water from the nozzles.  
**Only use air and not water.**

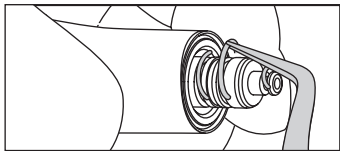
4) Remove the tip from the handpiece. If powder has settled on the O-ring, remove it and clean both the O-ring and the grooves around it well.

5) Remove the handpiece from the cord.

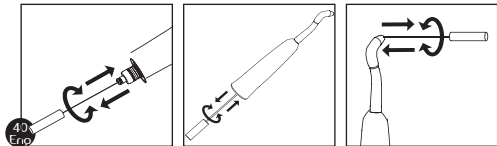6) Insert the nozzle needle into the points as shown in the figure and, with a rotating movement, remove the powder. Remove the needle and blow the cannula with compressed air.


**! WARNING!**

Do not force the wire into the duct, it may break inside the duct, interrupting the normal flow of water or air.

7) Unscrew the cap.

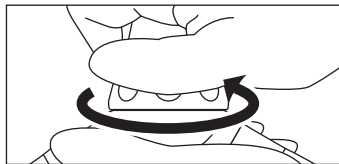Wash the parts under running water to remove any powder that has accumulated inside the container or on the cap.

8) Carefully dry all parts and reassemble them.


9) Clean the handpiece with disinfectant wipes.

## - Lubrication
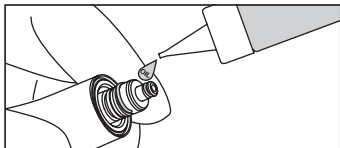
### **! WARNING!**

Failure to lubricate the O-rings on the handpiece side may cause damage to the coupling, with the consequent risk of impeding the rotation of the handpiece connection, water leakage etc.

- Dry the O-ring of the powder container well and place a small drop of oil on it, spreading it well with your fingers or other tool.

### **! WARNING!**

Lubricate only with Ivory Oil ref.OL101S

### **! WARNING!**

The tip is the only part of the Prophy-Silk handpiece that can be autoclaved or thermally disinfected.

Autoclave sterilisation in accordance with:EN 13060, EN ISO 17665-1

### **! WARNING!**

- Follow the steriliser manufacturer's instructions.
- Clean and disinfect before sterilisation. The presence of any coagulated blood residues on the external or internal surfaces may lead to the formation of rust.
- Make sure that the sterilised material is dry.

### Authorised sterilisation procedures

- 1) Seal the handpiece and its accessories in sterilisation bags conforming to EN 868-5.
- 2) Sterilise the bagged handpiece in an autoclave according to **EN ISO 17665-1** standard and respect the following conditions:
  - Class B steam sterilisation in autoclaves conforming to **EN 13060** standard. Sterilisation time at least 3.5 minutes at 135 °C.

3) Remove the bag from the autoclave..

### -Storage

- Keep the handpiece in the sterilisation bag until it is needed.
- Once sterilised, store the product in a dry and dust-free place, if possible with a low microbial charge

### CAUTION!

Respect the validity date for the sterilisation!

## Disposal

-  Sterilise the product and its accessories before disposing of them. Refer the disposal of the product  to authorised companies qualified to dispose of special industrial waste.

## Technical assistance – Ivory direct line

To request technical assistance, compile the on-line form available at: [www.ivory-dent.com](http://www.ivory-dent.com)

## Declaration of conformity

This medical device conforms to the requirements of European Directives 93/42 CEE (Class I).  
This conformity is expressed by the mark:

Additional documentation can be requested from:  
[ivory@ivory-dent.com](mailto:ivory@ivory-dent.com)

## Warranty

Ivory guarantees this product against defects of material and manufacture for a period of 24 months from the date of purchase. Ivory cannot be held responsible for defects and consequences caused by or attributable to normal wear, improper use, cleaning or maintenance, failure to follow operating and coupling instructions, lime scale or corrosion, contamination of the air or water supplies, chemical or electrical effects of an exceptional nature or not permitted by the instructions provided by Ivory or other manufacturers. Ivory expressly declines all responsibility for defects or consequences attributable to tampering with the product or modifications made by the customer or unauthorised third parties, or attributable to the use of non-original spare parts. In the event of justified claims, Ivory shall either repair the product or supply spare parts free of charge under the warranty. The warranty excludes all other claims, with particular reference to claims for compensation. In the case of delays, serious or wilful negligence, this exclusion is only valid provided no specific laws are contravened.

The warranty does not apply to lamps, glass and optic fibre conductors, glass or rubber parts, or the colour fastness of plastic materials or all parts subject to normal wear. Claims made under the warranty must be presented by compiling the on-line request "QRRMA" form and by accompanying the product with a copy of the purchase invoice.

The purchase invoice must clearly identify the dealer, date of purchase, code or model of the product and the production or serial number. Please refer also to the articles 17,18 and 19 in the "Terms and payment" section on [www.ivory-dent.com](http://www.ivory-dent.com)

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KaVo® is a registered trademark of Kaltenbach & Voigt GmbH & Co., Germany. Sirona® is a registered trademark of Sirona Dental Systems GmbH, Germany. Bien-Air® is a registered trademark of Bien-Air Dental S.A., Switzerland. NSK® is a registered trademark of NSK NAKANISHI INC., Japan. W&H® and Roto Quick® are registered trademarks of W&H Dentalwerk Bürmoos GmbH, Austria.